



medemagruppen



Reservedele/Spare parts MC 1122 Quantum

09-2022




Indhold	
Undervogn basic	4
Affjedring og svinghjul	6
Stænkskærm set	8
Centerhjul	10
Motor	12
Sædemodul 1122, med tilt funktion	14
Sædemodul 1122, el-hejs & tilt	16
Gardiner for MBL hejs og tilt	18
Vægtklodser, indre, komplet	20
Vægtklodser	22
Fenderhjul for vægtklodser	24
Fenderhjul, sæt MC 1122	26
Spinalus MC 1122 uden puder	28
Nakkestøtte Junior	30
Nakkestøtte voksen	32
Fodstøtte aktuator del	34
Monteringsbeslag f/fodstøtte	36
Fodstøtte, EL voksen	38
Fodstøtte, manuel voksen	40
Fodplade	42
Lægpude	44
Centralfodplade, voksen, EL	46
Fodplade centerdel	48
Stolpe inderdel	50
Centralfodplade, barn, EL	52
Centralfodplade, voksen manuel	54
Centralfodplade, Junior manuel	56
Artikulær fodplade - komplet øvre samling	58
Artikulær fodplade - Aktuatorer og afdækning	60
Artikulær fodplade - Elektronisk	62
Artikulær fodplade - 10-16" nedre forlænger med fodplade	64
Artikulær fodplade - 13-19" nedre forlænger med fodplade	65
Centermonteret fodplade - komplet samlet	66
AFP/Centermonteret fodplade - lægstøtter	67
Centermonteret fodplade - Pediatric	68
Centermonteret fodplade - Standard	70
Sving væk joystickholder (voksen)	72
Armlæn	74

Content	
Undercarriage basic	4
Suspension and pivot wheels	6
Mudguard set	8
Central Wheel	10
Motor	12
Seat module 1122 incl tilt function	14
Seat module MBL electric hoist and tilt	16
Lateral blind for MBL lift/tilt	18
Weight, Inner, complete	20
Weight brackets	22
Fender wheels for weights	24
Fender wheels, set MC 1122	26
Spinalus MC 1122 without cushions	28
Headrest Junior	30
Headrest adult	32
Footrest actuator part	34
Mounting bracket for footrest	36
Footrest, electric adult	38
Footrest, manual adult	40
Footplate	42
Calf rest	44
Central footplate, adult, electric	46
Footrest center console	48
Console innerpart	50
Central footplate, child, electric	52
Central footplate adult, manual	54
Central footplate, Junior manual	56
Articulating Foot Platform - Complete Upper Assembly	58
Articulating Foot Platform - Actuators and Shrouds	60
Articulating Foot Platform - Electronics	62
Articulating Foot Platform - 10-16" Lower Extensions with Footplates	64
Articulating Foot Platform - 13-19" Lower Extensions with Footplates	65
Center Mount Foot Platform - Complete Assemblies	66
AFP/Center Mount Foot Platform - Calf Pads	67
Center Mount Foot Platform - Pediatric	68
Center Mount Foot Platform - Standard	70
Swing-away joystick mount (adult)	72
Armrests	74



MC 1122 Quantum

Undervogn basic
Undercarriage basic

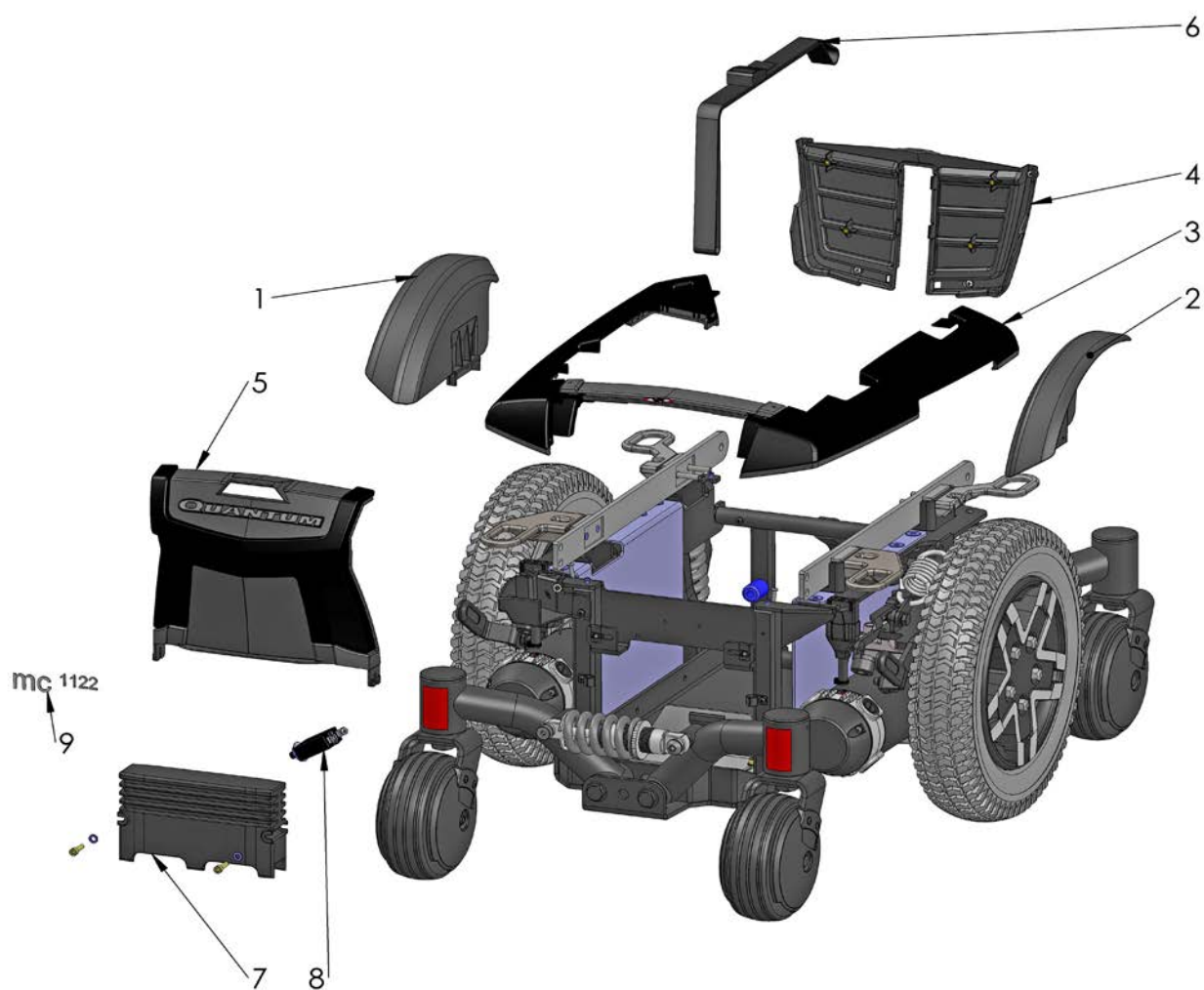


MC 1122 Quantum

Undervogn skærme og power modul
 Undercarriage covers and power module

Pos.	Del/part no.	Benævnelse	Part name	Antal/ quant.
1	2401-1238	Hjulskærm venstre MC 1122	Wheel cover, left, MC 1122	1
2	2401-1237	Hjulskærm højre MC 1122	Wheel cover, right, MC 1122	1
3	2401-1241	Centerskærm MC 1122	Central cover, MC 1122	1
4	2401-1239	Skærm front, MC 1122	Front cover, MC 1122	1
5	2401-1240	Skærm bag, MC 1122	Back cover, MC 1122	1
6	1505-1185	Batterirem	Battery strap	1
7	2405-1242	Power modul PMA til DX2	Power module PMA for DX2	1
8	2405-1109	Termosikring u/flysikring 70A	Thermal fuse no/ flight security 70A	1
9	2409-1017	Label sølv, MC 1122	Silver label, MC 1122	1

MC 1122 Quantum

Affjedring og svinghjul
Suspension and pivot wheels

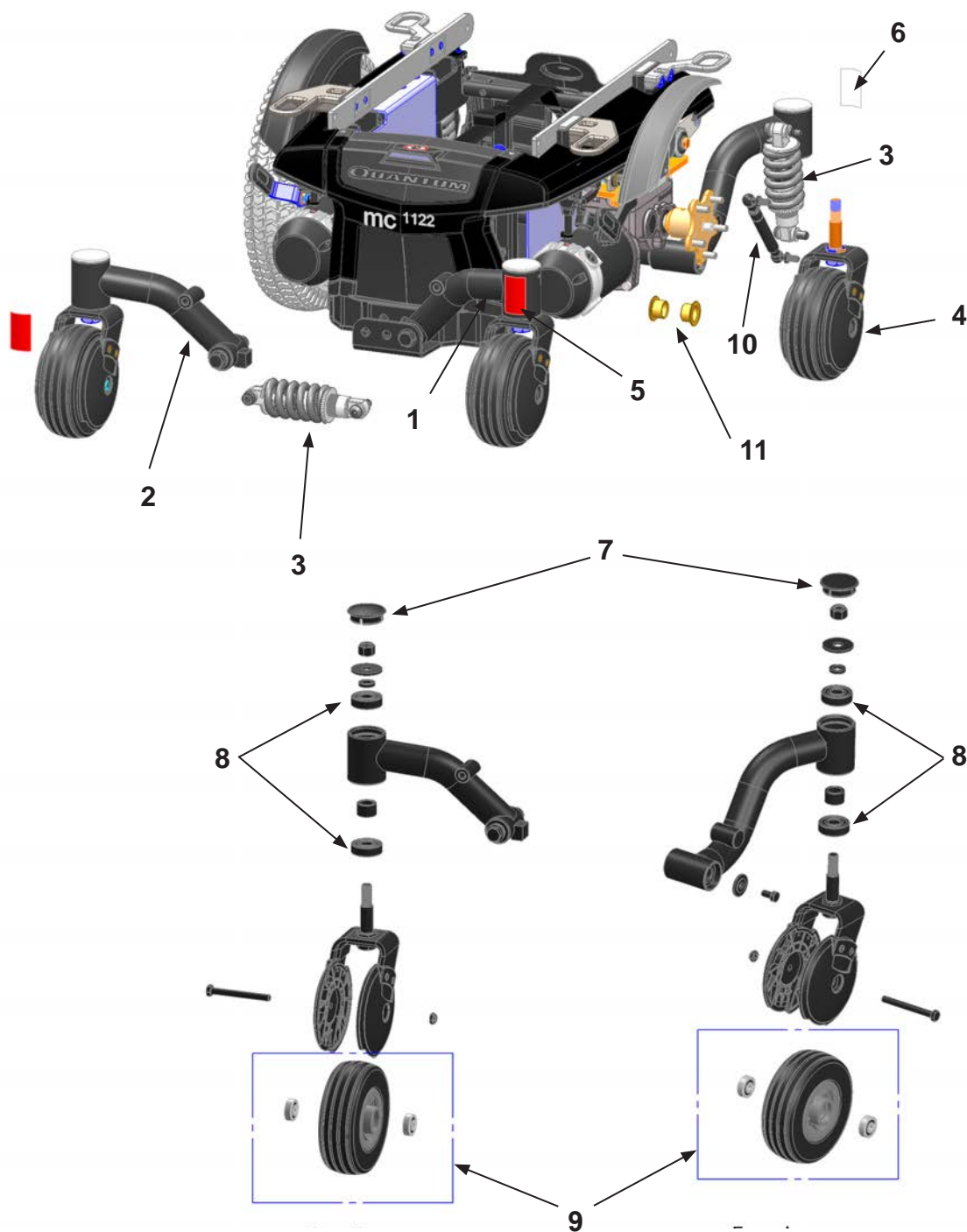

MC 1122 Quantum

Affjedring og svinghjul Suspension and pivot wheels


Pos.	Del/part no.	Benævnelse	Part name	Antal/quant.
1	2406-1210	Affjedring, bag m/6" svinghjul, højre	Suspension back, w/6" wheel, right	1
2	2406-1209	Affjedring, bag m/6" svinghjul, venstre	Suspension back, w/6" wheel, left	1
3	2406-1211	Fjeder til affjedring, MC 1122	Suspension spring back, MC 1122	1
4	2406-1212	Svinghjul m/gaffel	Pivot wheel w/fork	2
5	2409-1003	Reflekslabel rød 30x50	Reflective label red 30x50	2
6	2409-1004	Reflekslabel hvid 30x50	Reflective label white 30x50	2
7	2435-1027	Topdæksel til affjedring bag	Top cover for suspension back	2
8	2403-1162	Kugleleje Ø40/Ø16 x 12 mm	Bearing ball Ø40/Ø16 x 12 mm	4
9	2406-1213	Svinghjul m/lejer	Pivot wheel w/bearing balls	4
10	2406-1214	Støddæmper til svinghjul, 1122	Suspension for pivot wheels , 1122	1
11	2435-1028	Bøsning for svinghjulsarm, MC 1122	Bushing for pivotwheels arm, MC1122	2



medemagruppen

MC 1122 Quantum

Centerhjul
Central Wheel

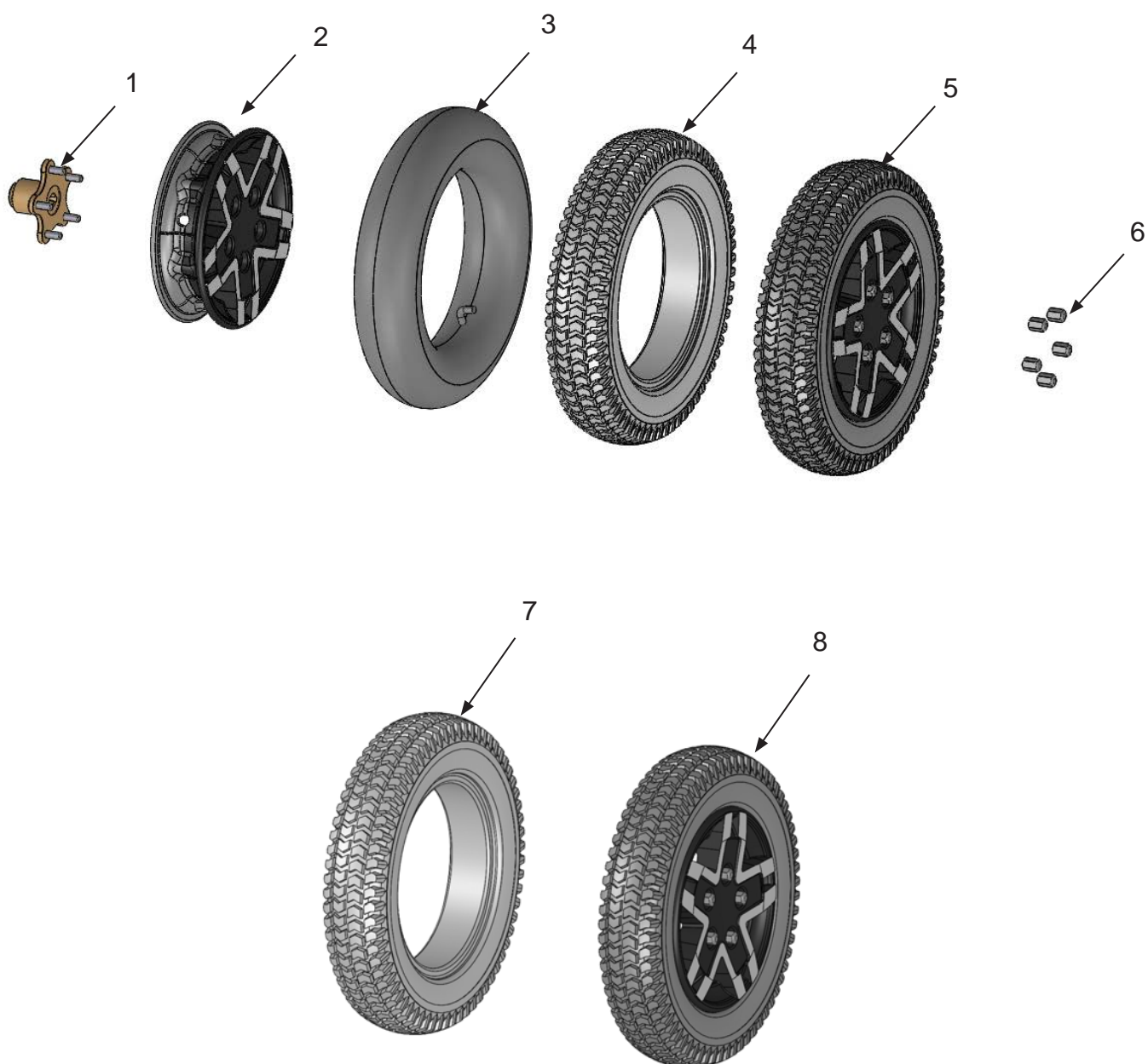


MC 1122 Quantum

Centerhjul
Central Wheel

Pos.	Del/part no.	Benævnelse	Part name	Antal/ quant.
1	2406-1329	Hjul nav, 1122	Wheel hub, 1122	1
2	2403-1149	Fælg komplet	Rim complete, 1122	1
3	2403-1110	Slange til trækjul 1120/1121/1122 (3.00-8)	Central hose 1120/1121/1122 (3.00-8)	1
4	2403-1107	Dæk, træk 1120/1122	Central tyre 1120/1122	1
5	2405-1236	Hjul, træk 1122, inkl fælg, luft	Central wheel 1122, pneumatic	1
6	2431-1010	Hjilmøtrikker 5 STK, 1122	Wheel nuts, 5 pcs.	1
7	2403-1148	Hjul, træk 1122, inkl. fælg, massiv	Central solid wheel 1122 with rim.	1
8	2403-1119	Dæk, træk 1120/1122, massiv	Central solid tyre 1120/1122.	1

MC 1122 Quantum

Motor
Motor

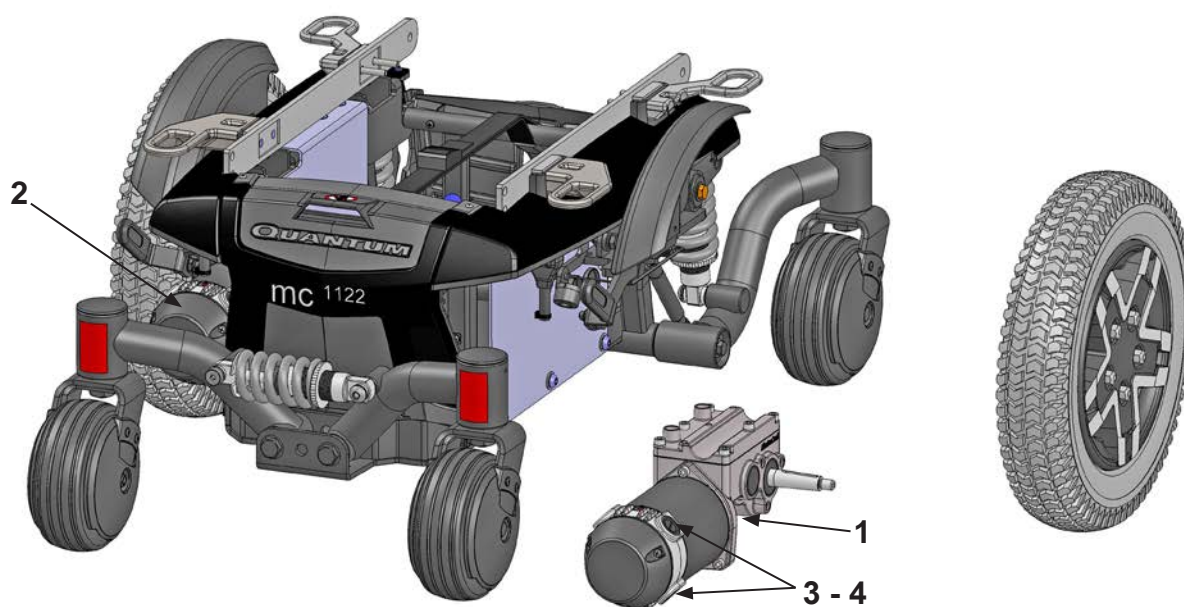

MC 1122 Quantum


Motor

Motor

Pos.	Del/part no.	Benævnelse	Part name	Antal/ quant.
1	2404-1237	Motor/gear, MC 1122 komplet højre	Motor/gear, MC 1122 complete right	1
2	2404-1238	Motor/gear, MC 1122 komplet venstre	Motor/gear, MC 1122 complete left	1
3	2404-1239	Kul til motor, MC 1122	Motor brush, MC 1122	8
4	2404-1236	Låg til kul, MC 1122	Lid for coal, MC 1122	8

MC 1122 Quantum

Sædemodul 1122, med tilt funktion
Seat module 1122 incl tilt function

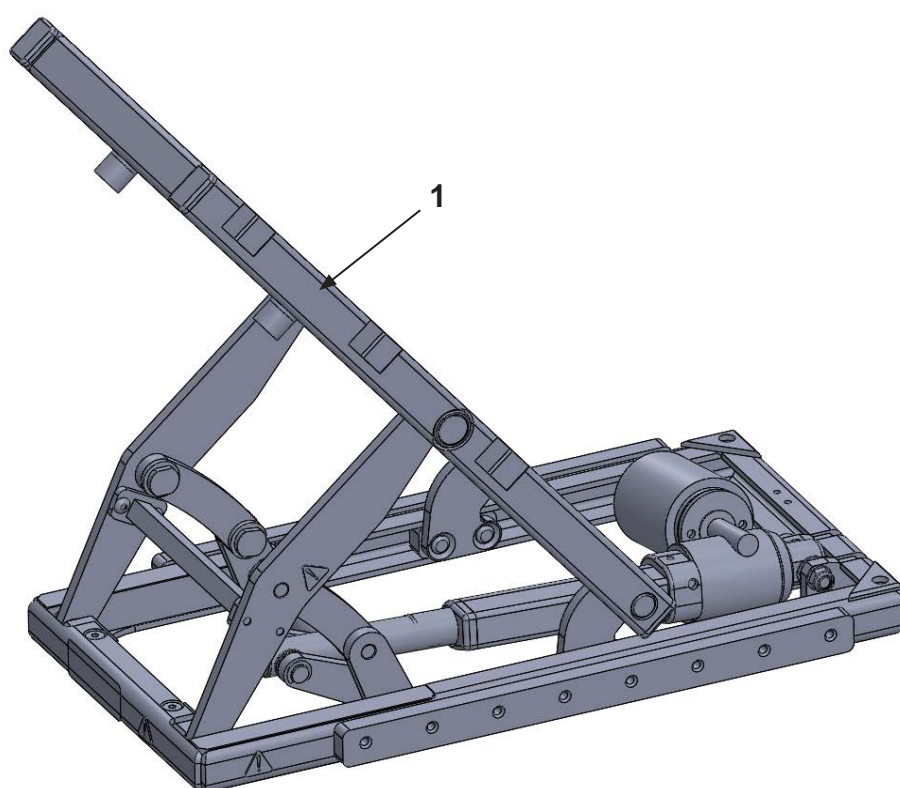


MC 1122 Quantum

Sædemodul 1122, med tilt funktion
Seat modul 1122 incl tilt function

Pos.	Del/part no.	Benævnelse	Part name	Antal/ quant.
1	2408-1216	Sædemodul, El-tilt - MBL	Seat module tilt - MBL	1

MC 1122 Quantum

Sædemodul 1122, el-hejs & tilt
Seat module MBL electric hoist and tilt

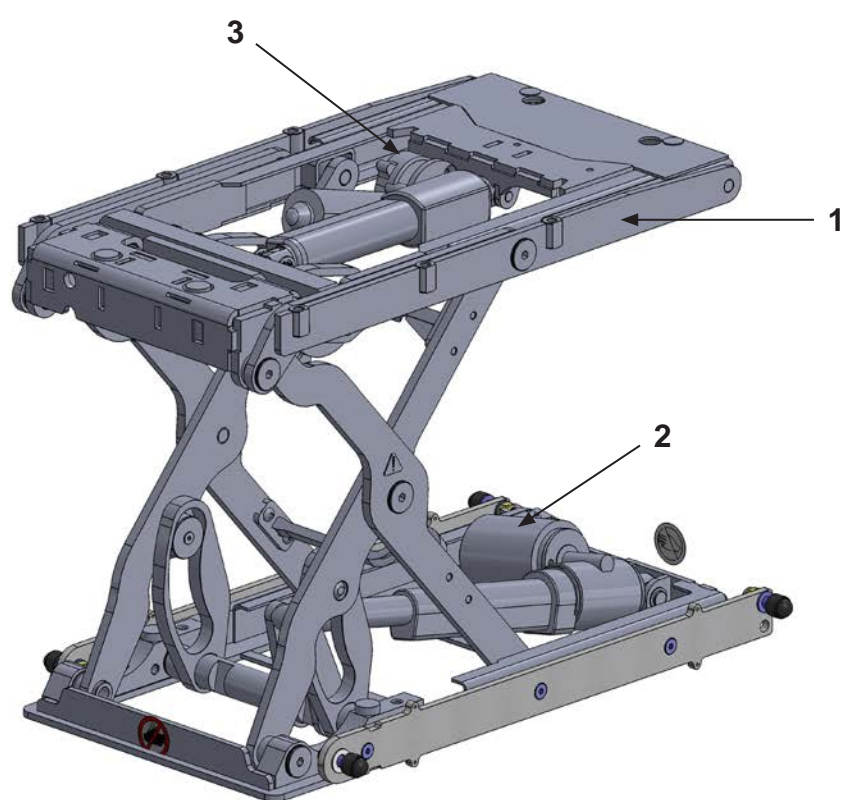


MC 1122 Quantum

Sædemodul 1122, el-hejs & tilt
 Seat module MBL electric hoist and tilt

Pos.	Del/part no.	Benævnelse	Part name	Antal/quant.
1	2408-1213	Sædemodul MBL el-hejs og tilt	Seat module MBL electric hoist and tilt	1
2	2404-1240	Hejs aktuator for MBL hejs og tilt (Centermål 104 mm)	Actuator for MBL hoist and tilt (Centre to centre 104 mm)	1
	2404-1487	Hejs aktuator for MBL hejs og tilt (Centermål 114 mm)	Actuator for MBL hoist and tilt (Centre to centre 114 mm)	1
3	2404-1241	Tilt aktuator for MBL hejs og tilt	Actuator for MBL hoist and tilt	1

MC 1122 Quantum

Gardiner for MBL hejs og tilt
Lateral blind for MBL lift/tilt.

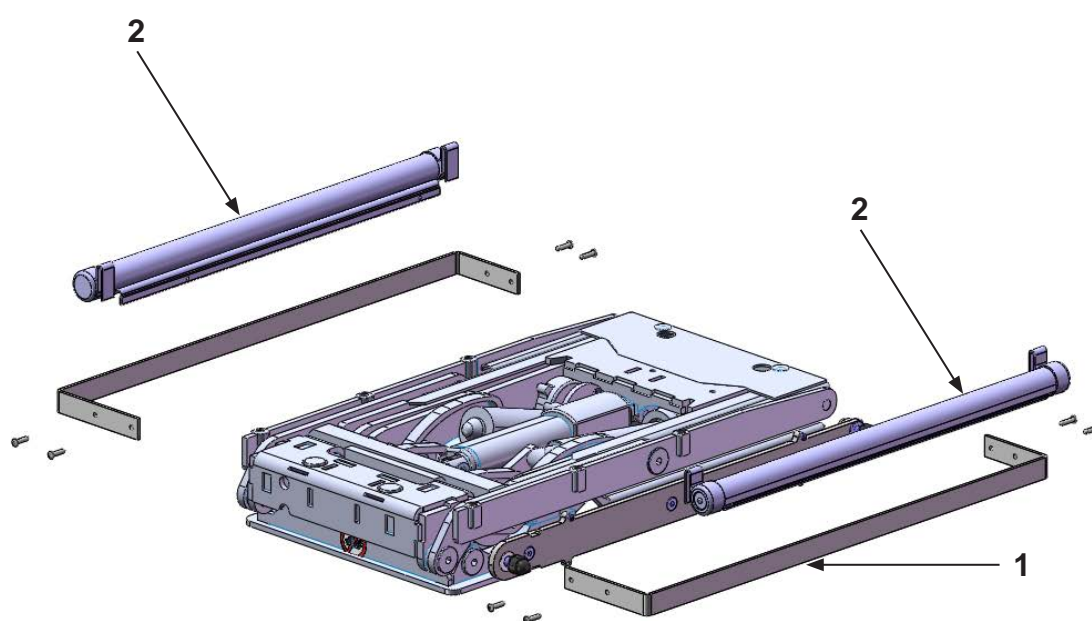

MC 1122 Quantum


CR-03004 Gardiner for MBL hejs og tilt

CR-03004 Lateral blind for MBL lift/tilt.

Pos.	Del/part no.	Benævnelse	Part name	Antal/quant.
1	2407-1098	Beslag for rullegarding	Bracket for lateral blind	2
2	2408-1215	Rullegardin højre/venstre	Lateral blind, right/left	1

MC 1122 Quantum

Vægtklodser, indre, komplet
Weight, Inner, complete

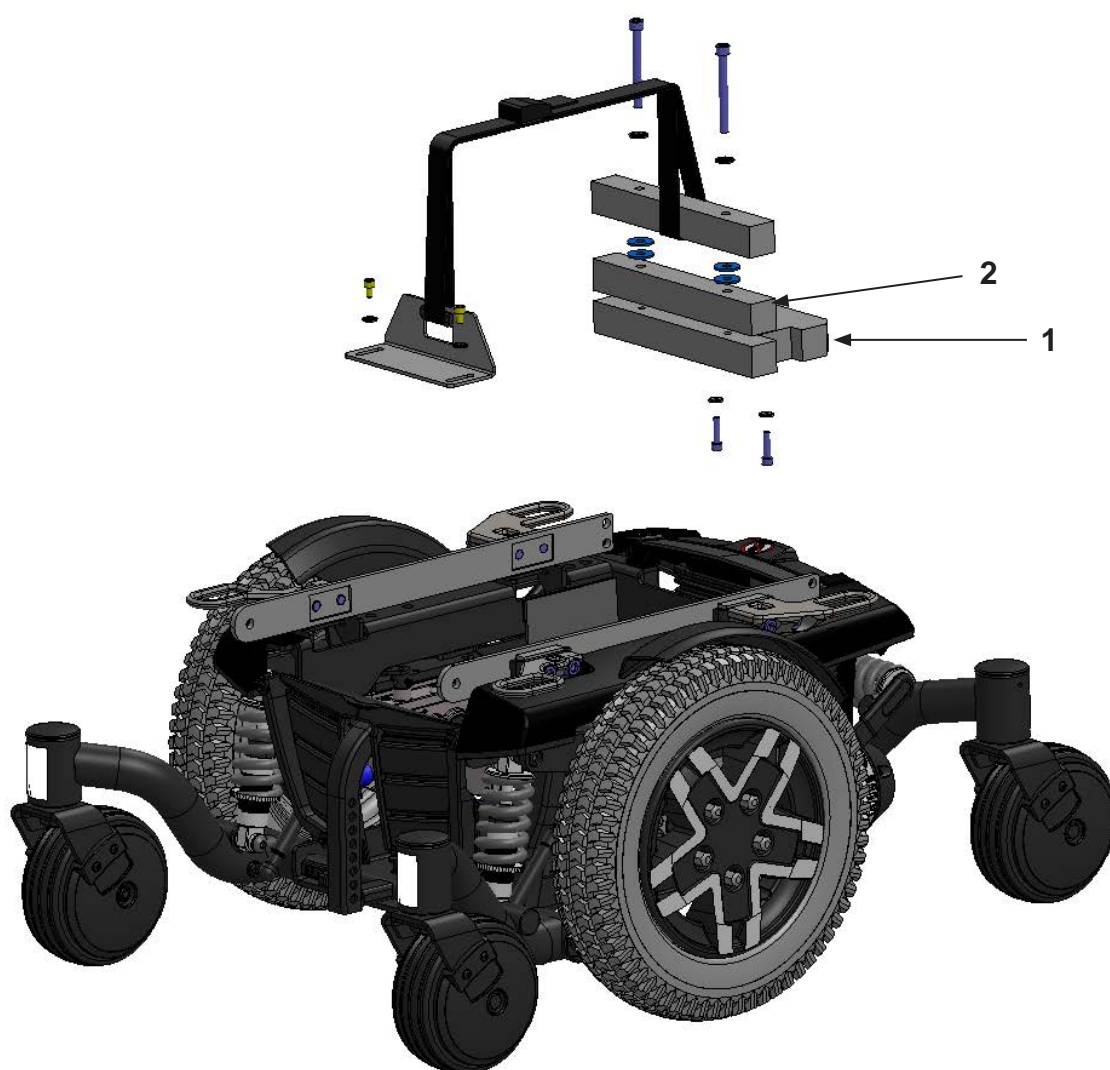

MC 1122 Quantum

CR-03056 Vægtklodser, indre, komplet

CR-03056 Weight, Inner, complete

Pos.	Del/part no.	Benævnelse	Part name	Antal/quant.
1	2406-1324	Vægtklods, bund, MC 1122	Weight, bottom MC 1122	1
2	2406-1325	Vægtklods 30x30x220 mm, MC 1122	Weight, 30 x 30 x 220 mm, MC 1122	2

MC 1122 Quantum

Vægtklodser
Weight brackets



MC 1122 Quantum


CR-03055 Vægtklodser

CR-03055 Weight brackets

Pos.	Del/part no.	Benævnelse	Part name	Antal/ quant.
1	2406-1326	Vægtklods for bagbom H, 1122	Weight for rear beam R, 1122	1
2	2406-1327	Vægtklods for bagbom V, 1122	Weight for rear beam L, 1122	1

MC 1122 Quantum

Fenderhjul for vægklodser
Fender wheels for weights

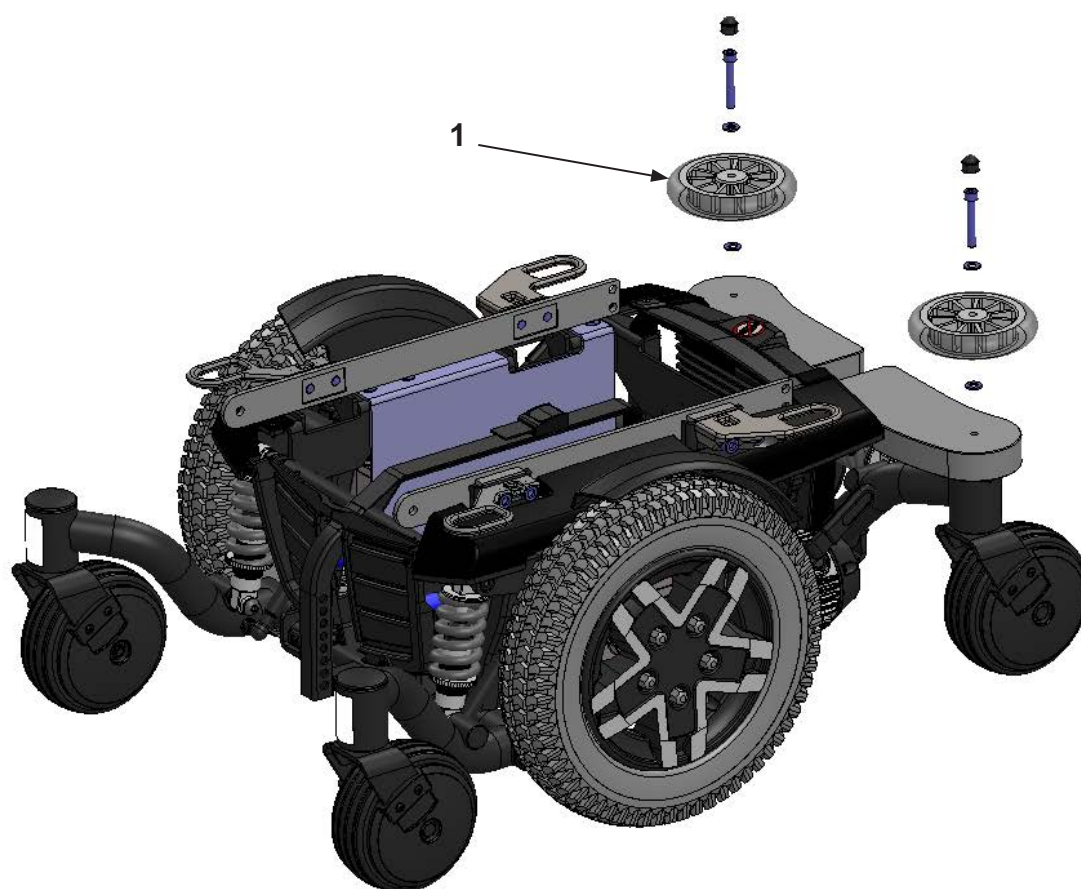

MC 1122 Quantum


CR-03062 Fenderhjul for vægklodser

CR-03062 Fender wheels for weights

Pos.	Del/part no.	Benævnelse	Part name	Antal/ quant.
1	2403-1121	Fenderhjul 125mm m/ klar bane	Fender wheel 125 mm with clear tread, Rolko	2

MC 1122 Quantum

Fenderhjul, sæt MC 1122
Fender wheels, set MC 1122

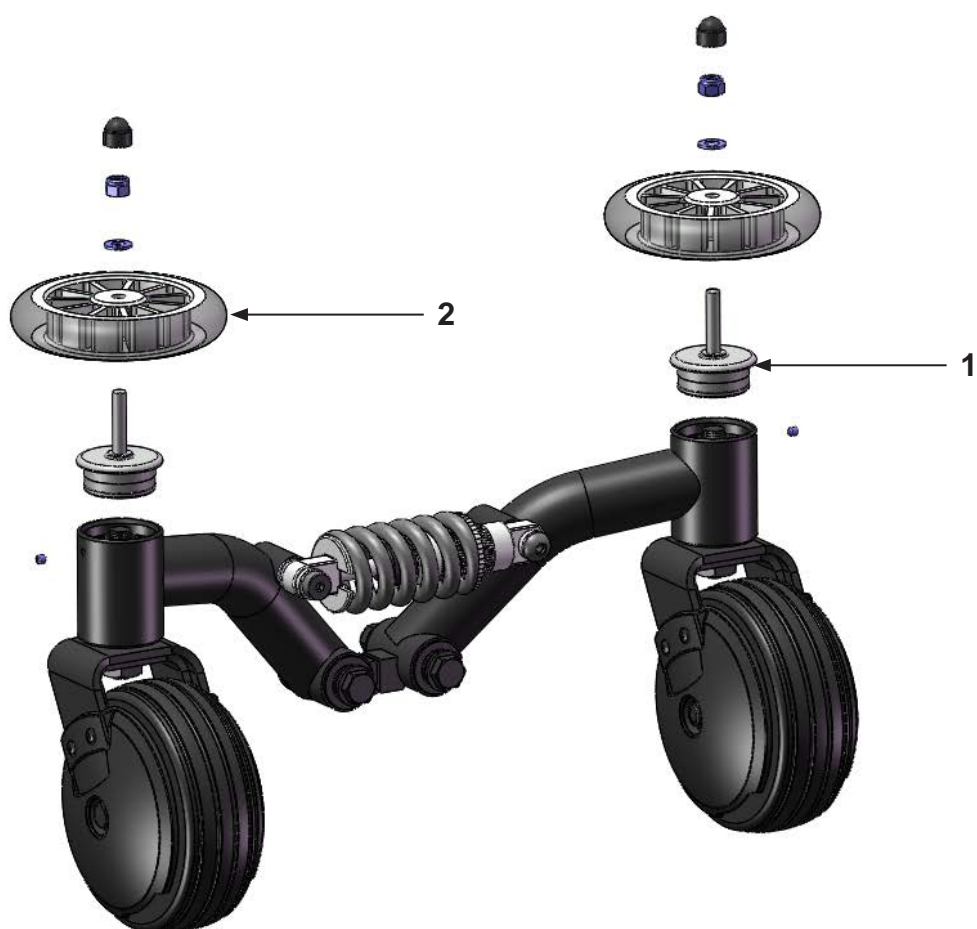

MC 1122 Quantum


CR-02809 Fenderhjul, sæt MC 1122

CR-02809 Fender wheels, set MC 1122

Pos.	Del/part no.	Benævnelse	Part name	Antal/ quant.
1	2406-1316	Holder for fenderhjul, MC 1122	Bracket for fender wheel, MC 1122	2
2	2403-1121	Fenderhjul 125 mm m/ klar bane	Fender wheel 125 mm with clear tread, Rolko	2

MC 1122 Quantum

Spinalus MC 1122 uden puder
 Spinalus MC 1122 without cushions

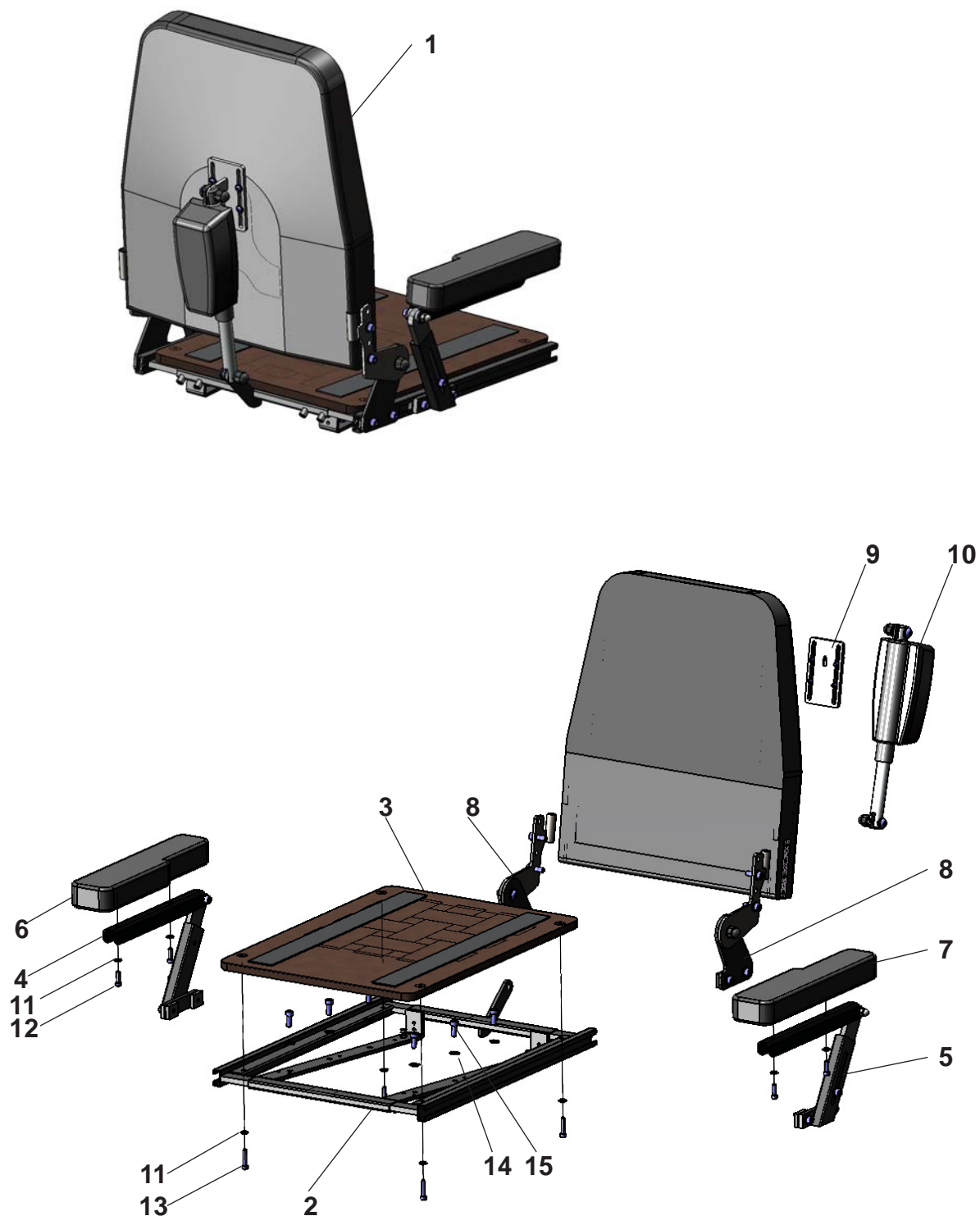

MC 1122 Quantum


Spinalus MC 1122 uden puder

Spinalus MC 1122 without cushions

Pos.	Del/part no.	Benævnelse	Part name	Antal/ quant.
1	2407-1083	Spinalus 40 cm - MC 1122 uden puder	Spinalus 40 cm - MC 1122 w/o cushions	1
	2407-1085	Spinalus 45 cm - MC 1122 uden puder	Spinalus 45 cm - MC 1122 w/o cushions	1
	2407-1087	Spinalus 50 cm - MC 1122 uden puder	Spinalus 50 cm - MC 1122 w/o cushions	1
	2407-1088	Spinalus 55 cm - MC 1122 uden puder	Spinalus 55 cm - MC 1122 w/o cushions	1
	2407-1163	Spinalus 60 cm - MC 1122 uden puder	Spinalus 60 cm - MC 1122 w/o cushions	1
2	1507-1332	Sæderamme 40-60 cm, Spinalus	Seat frame 40-60 cm, Spinalus	1
3	2407-1106	Træplade komplet til sæde 40 cm	Wooden plate complete for seat 40 cm	1
	1507-1176	Træplade komplet til sæde 45 cm	Wooden plate complete for seat 45 cm	1
	2407-1108	Træplade komplet til sæde 50 cm	Wooden plate complete for seat 50 cm	1
	2407-1109	Træplade komplet til sæde 55 cm	Wooden plate complete for seat 55 cm	1
	2407-1110	Træplade komplet til sæde 60 cm	Wooden plate complete for seat 60 cm	1
4	1507-1169	Armlænsholder lav voksen, højre	Armrest mount low adult, right	1
5	1507-1170	Armlænsholder lav voksen, venstre	Armrest mount low adult, left	1
6	2406-1136	Armlæns pude læder, 7 x 35 cm, højre	Armrest cushion leather, 7 x 35 cm, right	1
7	2406-1137	Armlæns pude læder, 7 x 35 cm, venstre	Armrest cushion leather, 7 x 35 cm, left	1
8	2418-1043	Rygforskydning (H+V) sæt, MBL	Backrest displacement (H + V) set, MBL	1
9	2407-1059	Rygbeslag MBL	Backrest bracket MBL	1
10	2404-1193	Aktuator 110 mm, komplet	Actuator 110 mm, complete	1
11	8031-1124	M6 Rip-lock sikringssskive A2	M6 Rip-lock lock washer A2	8
12	8031-1012	M6 x 20 ch A2	M6 x 20 ch A2	4
13	8031-1268	M6 x 30 ch A2	M6 x 30 ch A2	4
14	8005-1012	M8 Rip-lock sikringssskive A2	M8 Rip-lock lock washer A2	6
15	8031-1087	M8 x 20 ch A2	M8 x 20 ch A2	6

MC 1122 Quantum

Nakkestøtte Junior
Headrest Junior

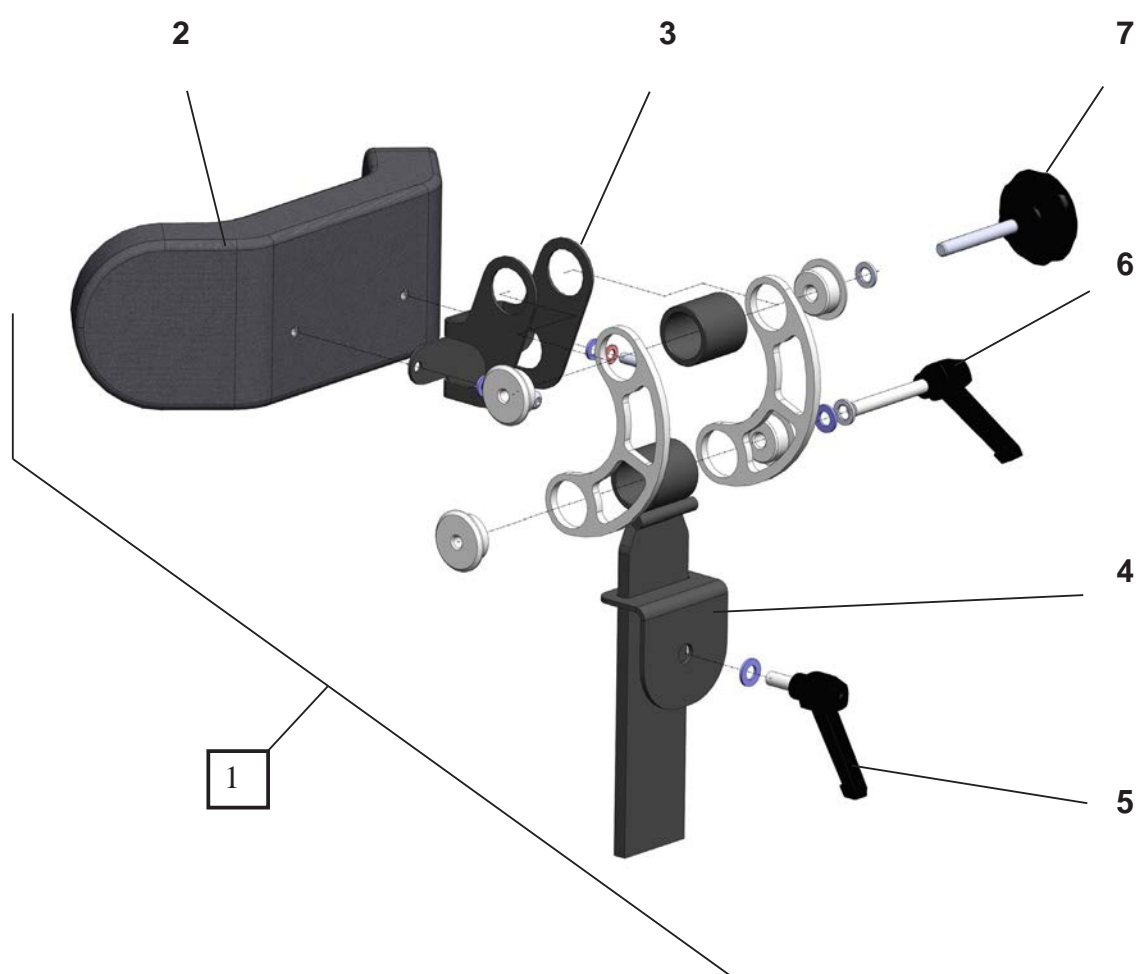

MC 1122 Quantum


Nakkestøtte Junior

Headrest Junior

Pos	Del/part no.	Beskrivelse	Part name	Antal/ quant.
1	2413-1078	Nakkestøtte m/justerbar pude, junior, stof	Headrest w/ ajustable pad, junior, fabric	1
1	2413-1079	Nakkestøtte m/justerbar pude, junior, PU	Headrest w/ ajustable pad, junior, PU	1
2	2413-1086	Nakkepude - justerbar, junior (stof)	Headrest - ajustable, junior (fabric)	1
2	2413-1088	Nakkepude- justerbar, junior (PU)	Headrest - ajustable, junior (PU)	1
3	1507-1328	U-beslag for nakkestøtte	U-bracket for headrest	1
4	2406-1074	Vinkel beslag til nakkestøtte	Angle bracket for headrest	1
5	2435-1015	Ø8x20 Kipspænde	Ø8x20 hand screw	1
6	8031-1243	Ø8x60 Kipspænde	Ø8x60 hand screw	1
7	8031-1264	Håndgreb med M8 x 60 gevind	Hand Lever w/ M8 x 60 thread	1

MC 1122 Quantum

Nakkestøtte voksen
Headrest adult

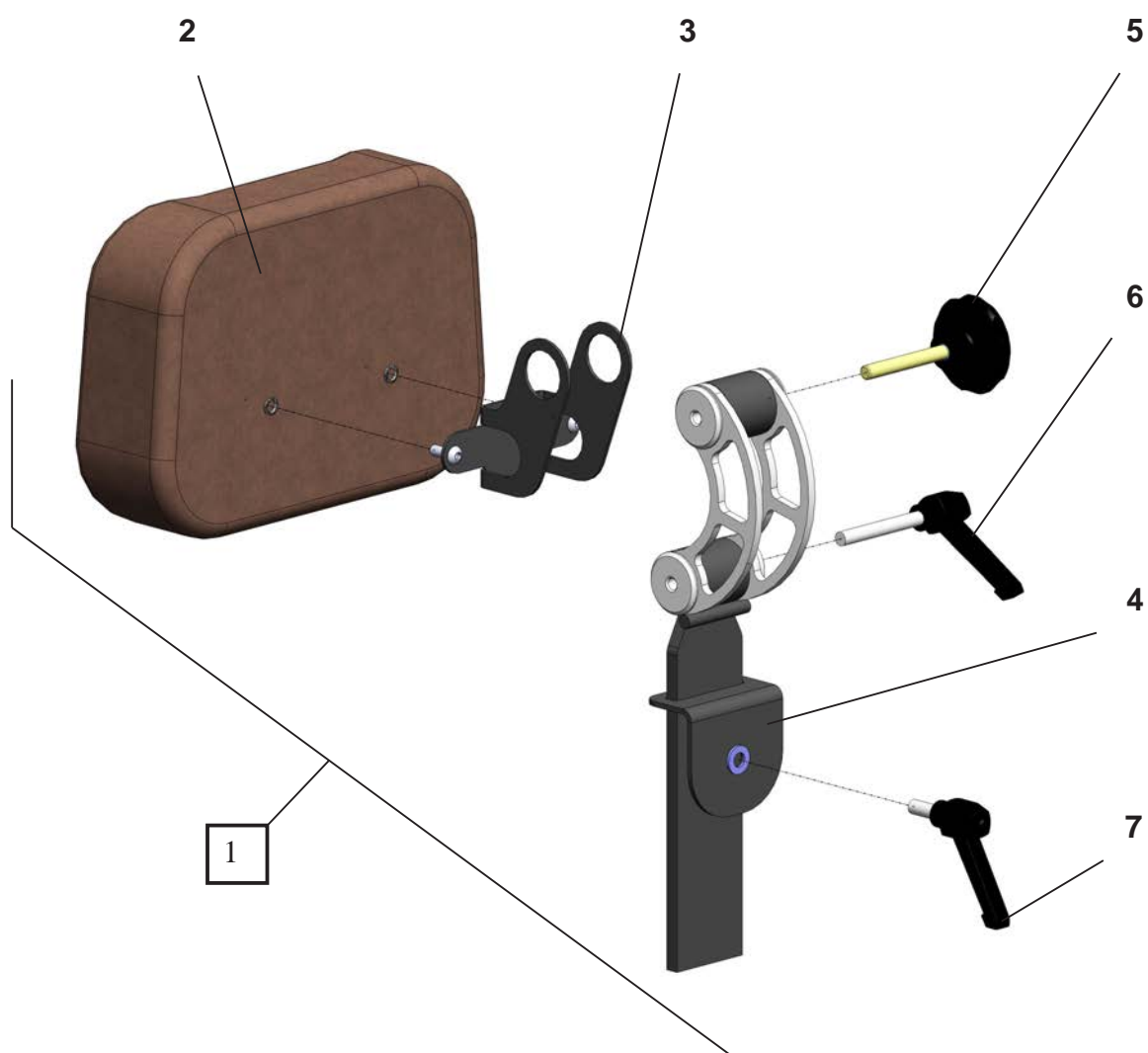

MC 1122 Quantum


Nakkestøtte voksen
 Headrest adult

Pos	Del/part no.	Beskrivelse	Part name	Antal/ quant.
1	2413-1076	Nakkestøtte m/justerbar pude, voksen, stof	Headrest w/ adjustable pad, adult, fabric	1
1	2413-1077	Nakkestøtte m/justerbar pude, voksen PU	Headrest w/ adjustable pad, adult PU	1
2	2413-1085	Nakkepude - justerbar, voksen, stof	Headrest - adjustable, adult, fabric	1
2	2413-1087	Nakkepude - justerbar, voksen, PU	Headrest - adjustable, adult, PU	1
3	1507-1328	U-beslag for nakkestøtte	U-bracket for headrest	1
4	2406-1074	Vinkelbeslag til nakkestøtte	Angle bracket for headrest	1
5	8031-1264	Håndgreb med M8x60 gevind	Hand Lever w/ M8 x 60 thread	1
6	8031-1243	Ø8x60 Kipspænde	Ø8x60 hand screw	1
7	2435-1015	Ø8x20 Kipspænde	Ø8x20 hand screw	1



MC 1122 Quantum

Fodstøtte aktuator del
Footrest actuator part

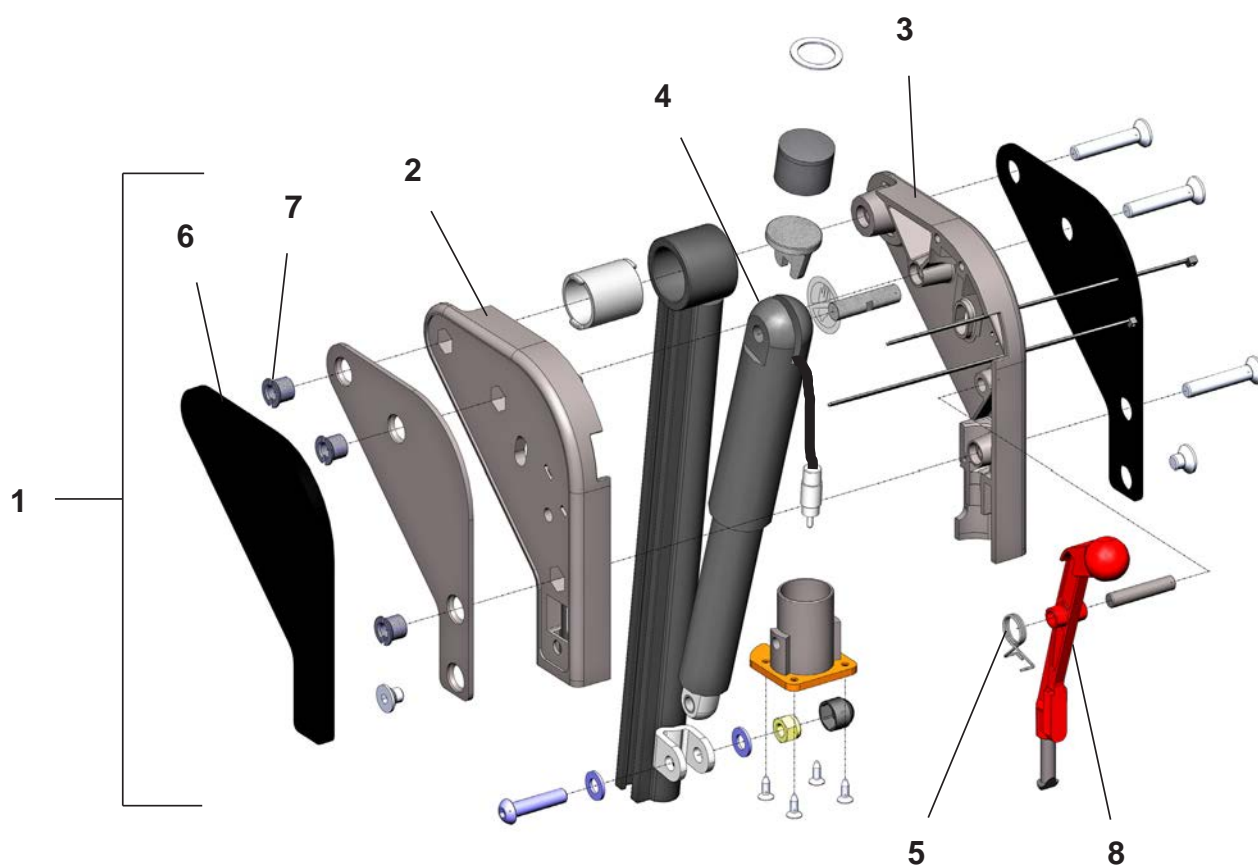

MC 1122 Quantum


Fodstøtte aktuator del

Footrest actuator part

Pos	Del/part no.	Beskrivelse	Part name	Antal/ quant.
1	2408-1166	Fodstøtte aktuator del højre	Footrest actuator part right	1
1	2408-1167	Fodstøtte aktuator del venstre	Footrest actuator part left	1
2	2408-1081	Benstøttehoved venstre	Legrest head left	1
3	2408-1082	Benstøttehoved højre	Legrest head right	1
4	2408-1182	Aktuator del elbenstøtte komplet	Actuator part electric legrest complete	1
5	2408-1085	Fjeder for lås benstøtte	Spring for lock, legrest	1
6	2408-1178	Knæled beskytter, højre	Knee protector, right	1
6	2408-1179	Knæled beskytter, venstre	Knee protector, left	1
7	2408-1088	M8 møtrik special	M8 special nut	3
8	2408-1185	Låseenhed benstøtte	Locking device for legrest	1

MC 1122 Quantum

Monteringsbeslag f/fodstøtte
Mounting bracket for footrest

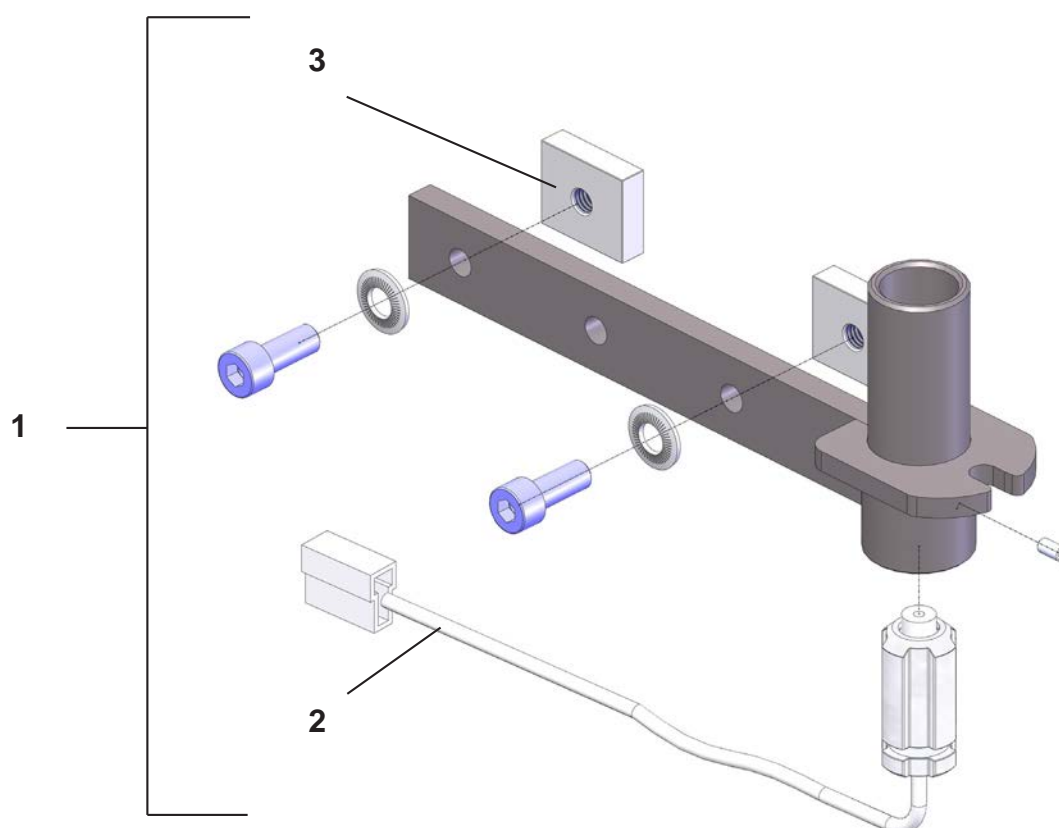


MC 1122 Quantum

Monteringsbeslag f/fodstøtte
Mounting bracket for footrest

Pos	Del/part no.	Beskrivelse	Part name	Antal/ quant.
1	2408-1160	Monteringsbeslag for fodstøtte højre	Mounting bracket for footrest right	1
1	2408-1161	Monteringsbeslag for fodstøtte venstre	Mounting bracket for footrest left	1
2	2408-1181	Stik benstøtte med ledning komplet	Plug for legrest complete w/ cable	1
3	1506-1149	C-skinne beslag	C-rail mounting	2

MC 1122 Quantum

Fodstøtte, EL voksen
Footrest, electric adult

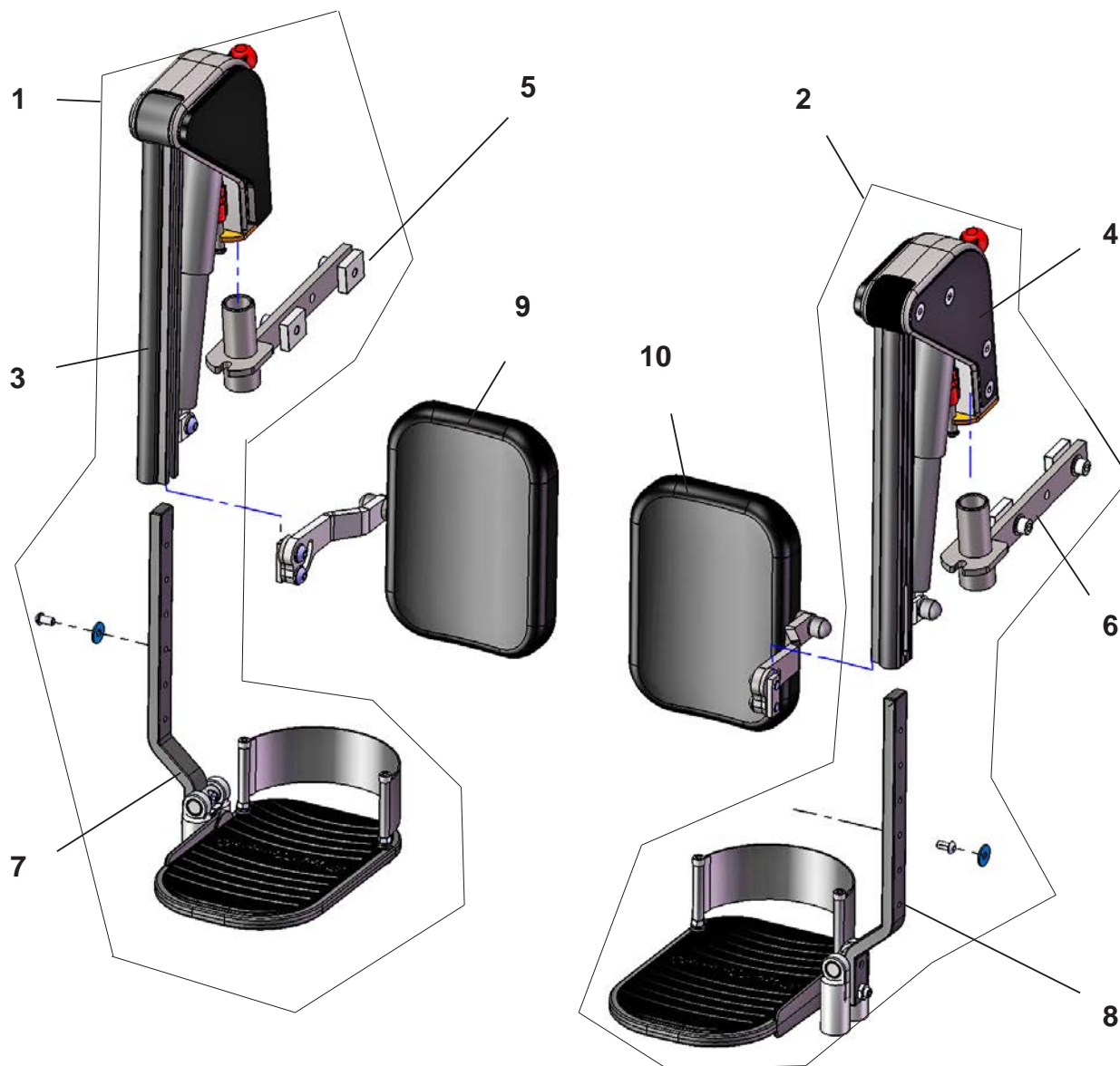


MC 1122 Quantum

Fodstøtte, EL voksen
 Footrest, electric adult

Pos	Del/part no.	Beskrivelse	Part name	Antal/ quant.
1	2408-1152	Fodstøtte EL voksen, højre, SB40-45	Footrest electric adult, right, SB40-45	1
2	2408-1156	Fodstøtte EL voksen, venstre, SB40-45	Footrest electric adult, left, SB40-45	1
3	2408-1166	Fodstøtte actuator del, højre	Footrest actuator part, right	1
4	2408-1167	Fodstøtte actuator del, venstre	Footrest actuator part, left	1
5	2408-1160	Monteringsbeslag f/fodstøtte, højre EL.	Mounting Bracket f/footrest, right electric.	1
6	2408-1161	Monteringsbeslag f/fodstøtte, venstre EL.	Mounting Bracket f/footrest, left electric.	1
7	2408-1169	Fodplade højre, SB40-45	Footplate right, SB40-45	1
7	2408-1192	Fodplade højre, SB50+	Footplate right, SB50+	1
7	2408-1168	Fodplade højre, SB35	Footplate right, SB35	1
8	2408-1172	Fodplade venstre, SB40-45	Footplate left, SB40-45	1
8	2408-1193	Fodplade venstre, SB50+	Footplate left, SB50+	1
8	2408-1171	Fodplade venstre, SB35	Footplate left, SB35	1
9	2408-1163	Lægpude højre komplet, SB40-45	Calf rest right complete, SB40-45	1
9	2408-1199	Lægpude højre komplet, SB50+	Calf rest right complete, SB50+	1
10	2408-1165	Lægpude venstre komplet, SB40-45	Calf rest left complete, SB40-45	1
10	2408-1200	Lægpude venstre komplet, SB50+	Calf rest left complete, SB50+	1

MC 1122 Quantum

Fodstøtte, manuel voksen
Footrest, manual adult

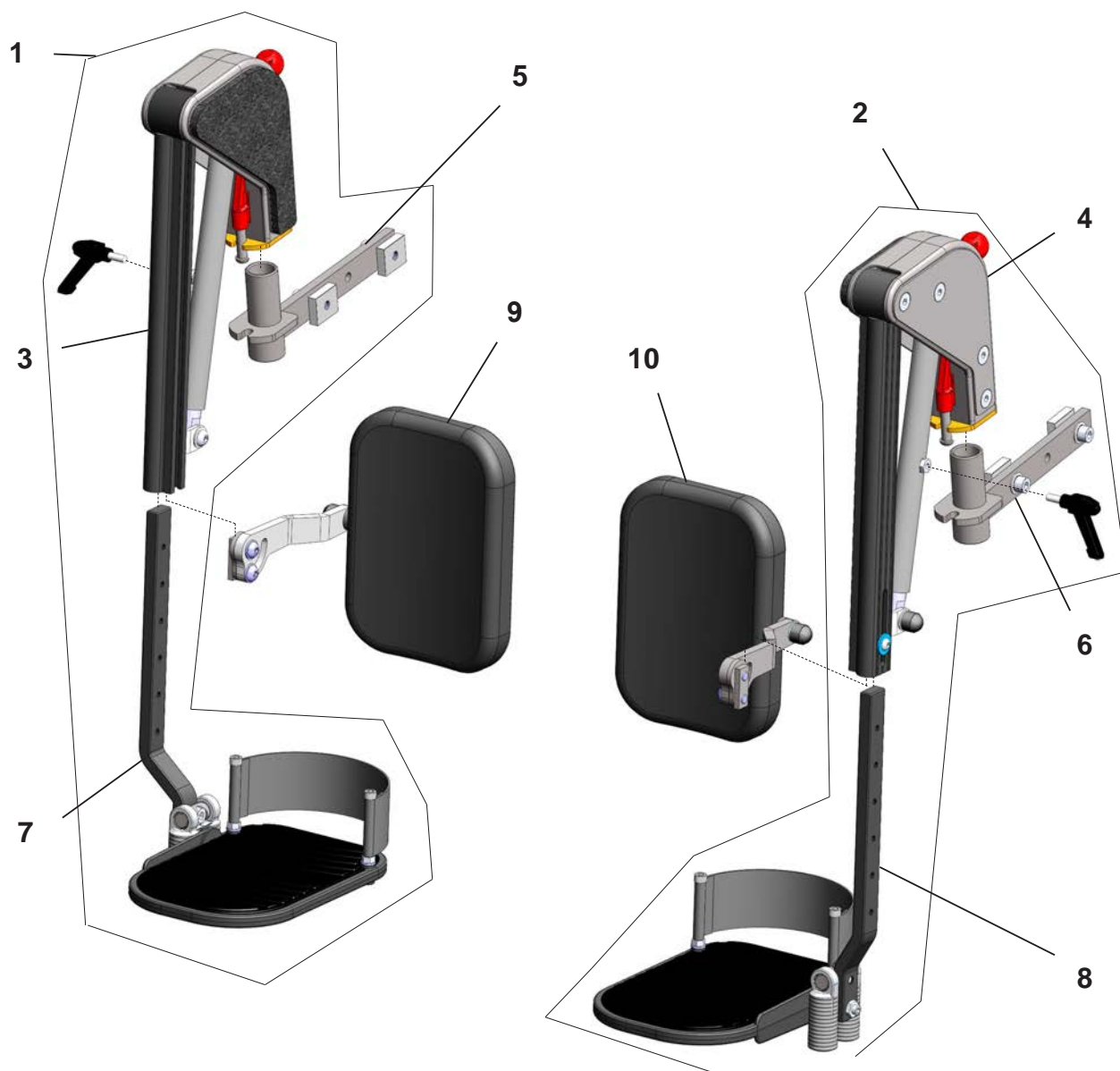

MC 1122 Quantum


Fodstøtte, manuel voksen

Footrest, manual adult

Pos	Del/part no.	Beskrivelse	Part name	Antal/ quant.
1	2408-1229	Fodstøtte manuel voksen, højre, SB40-45	Footrest manual adult, right, SB40-45	1
2	2408-1230	Fodstøtte manuel voksen, venstre, SB40-45	Footrest manual adult, left, SB40-45	1
3	2408-1234	Fodstøtte toppart, højre manuel	Footrest top part, right manual	1
4	2408-1235	Fodstøtte toppart, venstre manuel	Footrest top part, left manual	1
5	2408-1160	Monteringsbeslag f/fodstøtte, højre EL.	Mounting Bracket f/footrest, right	1
6	2408-1161	Monteringsbeslag f/fodstøtte, venstre EL.	Mounting Bracket f/footrest, left	1
7	2408-1169	Fodplade højre, SB40-45	Footplate right, SB40-45	1
7	2408-1192	Fodplade højre, SB50+	Footplate right, SB50+	1
7	2408-1168	Fodplade højre, SB35	Footplate right, SB35	1
8	2408-1172	Fodplade venstre, SB40-45	Footplate left, SB40-45	1
8	2408-1193	Fodplade venstre, SB50+	Footplate left, SB50+	1
8	2408-1171	Fodplade venstre, SB35	Footplate left, SB35	1
9	2408-1163	Lægpude højre komplet, SB40-45	Calf rest right complete, SB40-45	1
9	2408-1199	Lægpude højre komplet, SB50+	Calf rest right complete, SB50+	1
10	2408-1165	Lægpude venstre komplet, SB40-45	Calf rest left complete, SB40-45	1
10	2408-1200	Lægpude venstre komplet, SB50+	Calf rest left complete, SB50+	1

MC 1122 Quantum

Fodplade
Footplate

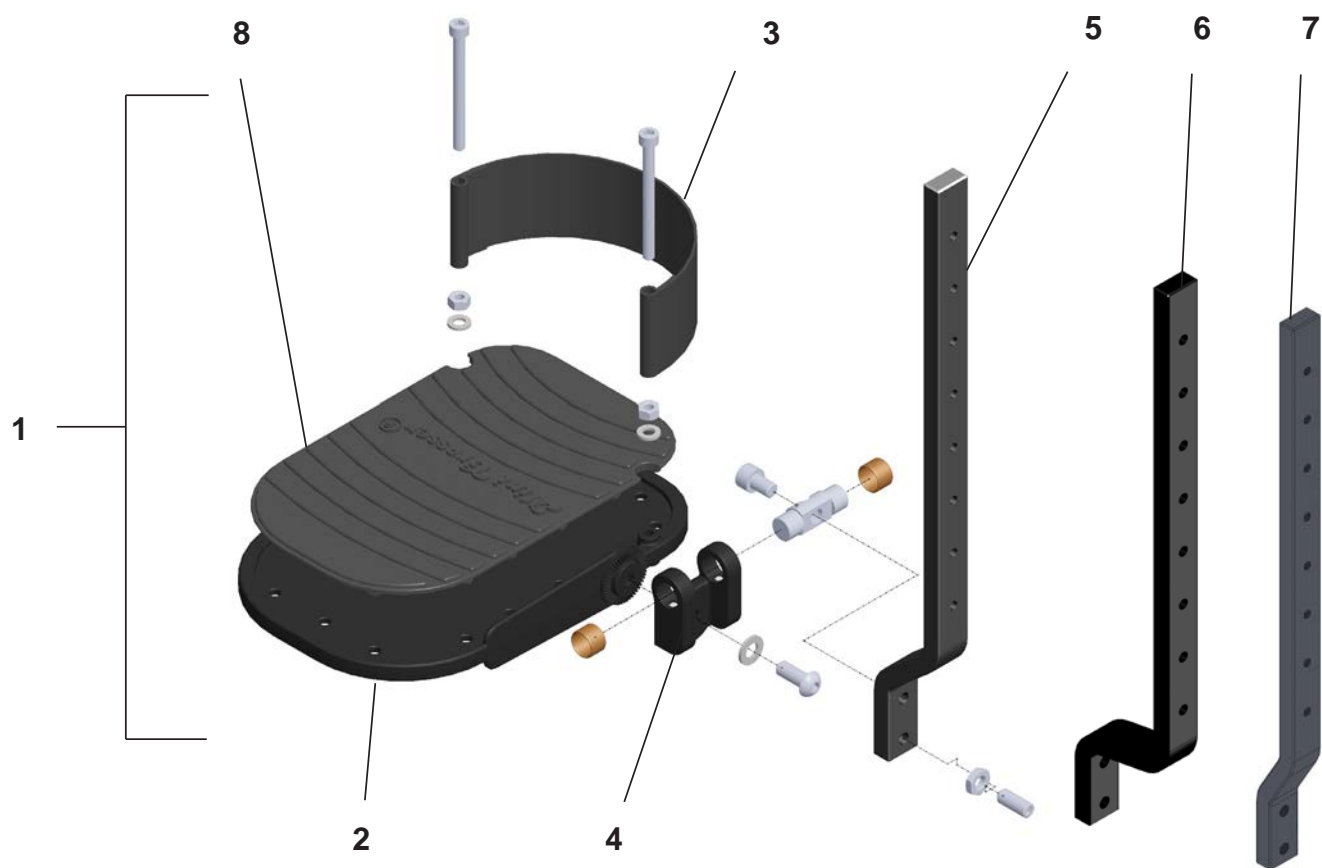

MC 1122 Quantum



Fodplade

Footplate

Pos	Del/part no.	Beskrivelse	Part name	Antal/ quant.
1	2408-1169	Fodplade højre SB40-45	Footplate right SB40-45	1
1	2408-1172	Fodplade venstre SB40-45	Footplate left SB40-45	1
1	2408-1168	Fodplade højre SB35	Footplate right SB35	1
1	2408-1171	Fodplade venstre SB35	Footplate left SB35	1
1	2408-1192	Fodplade højre SB50+	Footplate right SB50+	1
1	2408-1193	Fodplade venstre SB50+	Footplate left SB50+	1
1	2408-1176	Fodplade junior / barn højre	Footplate junior / child right	1
1	2408-1177	Fodplade junior / barn venstre	Footplate junior / child left	1
2	2408-1079	Fodplade aluminium	Footrest aluminum	1
3	2408-1187	Hælstrop alu fodplade	Heel strap alu footplate	1
4	2408-1208	Emne vinkling fodplade	Hinge angulation, footplate	1
5	2408-1180	Profil underdel el-benstøtte SB40-45	Profile lower part electric-legrest SB40-45	1
6	2408-1201	Profil underdel el-benstøtte SB50+	Profile lower part electric-legrest SB50+	1
7	2408-1175	Profil underdel el-benstøtte SB35	Profile lower part electric-legrest SB35	1
8	2408-1174	Måtte alufodplade	Mat aluminium footplate	1

MC 1122 Quantum

Lægpude
Calf rest

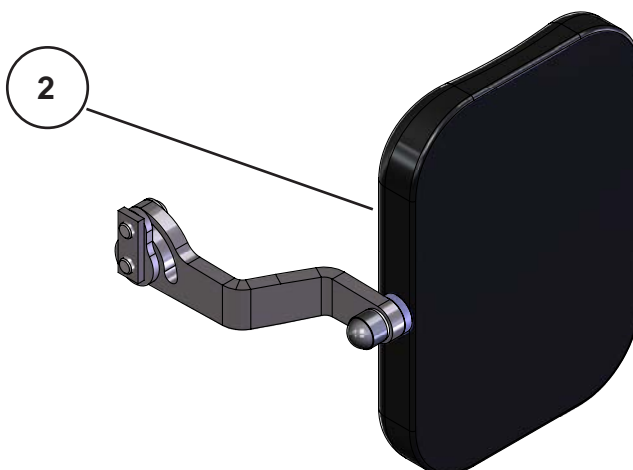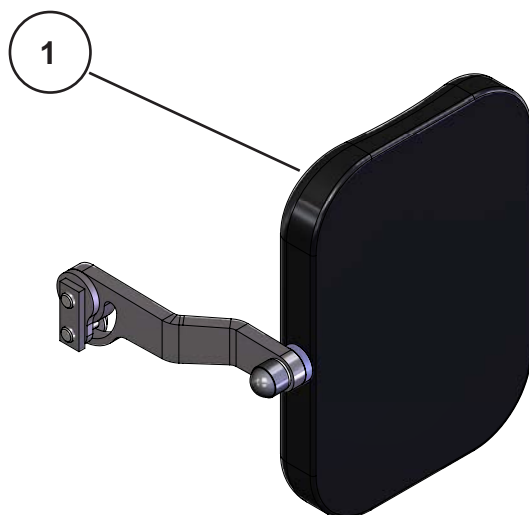


MC 1122 Quantum

Lægpude
Calf rest

Pos	Del/part no.	Beskrivelse	Part name	Antal/ quant.
1	2408-1163	Lægpude højre komplet SB40-45	Calf rest right complete SB40-45	1
1	2408-1165	Lægpude venstre komplet SB40-45	Calf rest left complete SB40-45	1
2	2408-1199	Lægpude højre komplet SB50+	Calf rest right complete SB50+	1
2	2408-1200	Lægpude venstre komplet SB50+	Calf rest left complete SB50+	1

MC 1122 Quantum

Centralfodplade, voksen, EL.
Central footplate, adult, electric

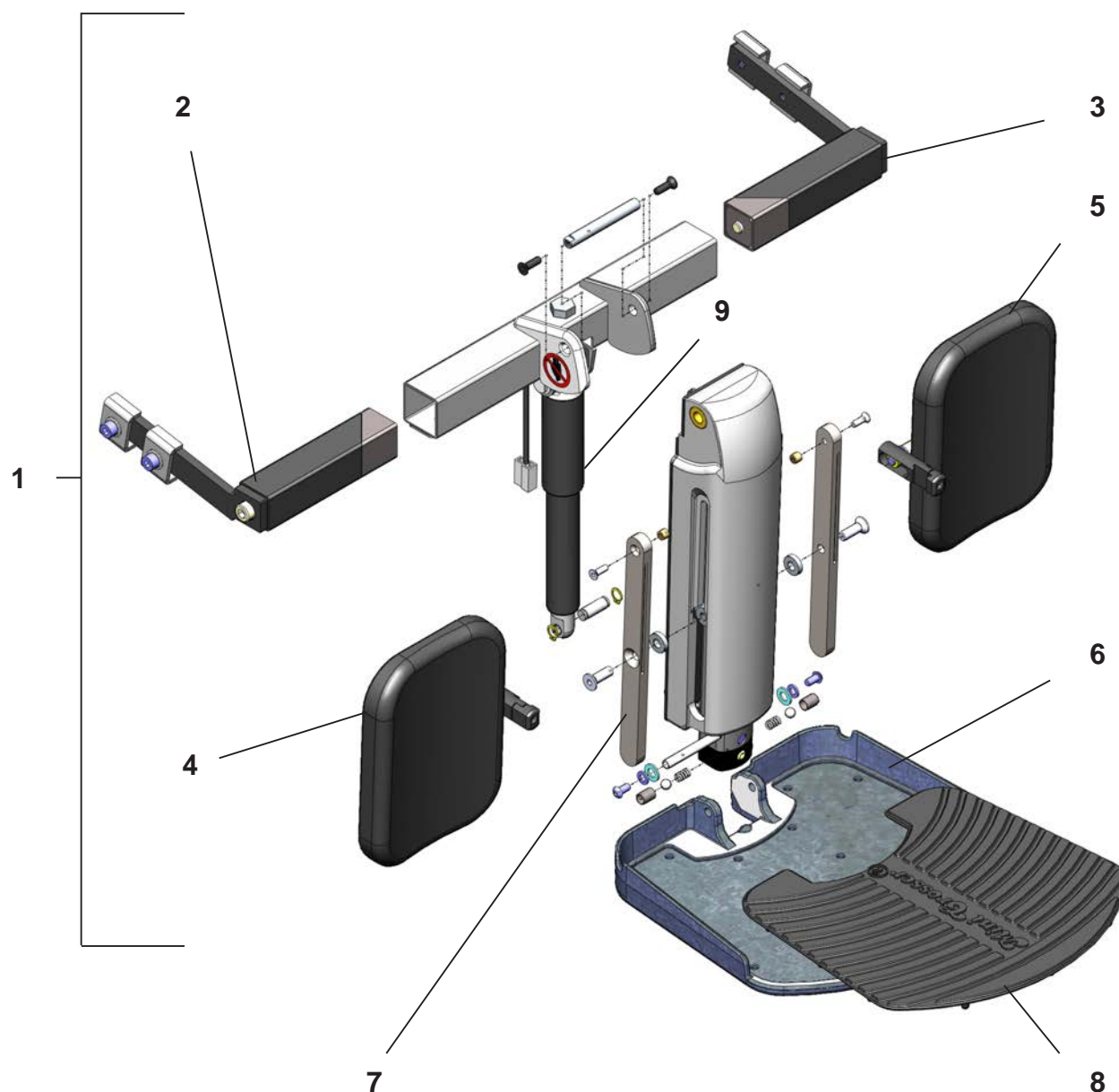


MC 1122 Quantum

Centralfodplade, voksen, EL.
 Central footplate, adult, electric

Pos	Del/part no.	Beskrivelse	Part name	Antal/ quant.
1	2406-1159	Fodbræt t/ voksen (ud-ind/op-ned)	Footplate for adult (out-in/up-down)	1
2	2408-1204	Torsionsled komplet voksen, højre	Torque joint complete adult, right	1
3	2408-1205	Torsionsled komplet voksen, venstre	Torque joint complete adult, left	1
4	2413-1093	Lægpude II komplet, højre	Calf rest II complete, right	1
5	2413-1095	Lægpude II komplet, venstre	Calf rest II complete, left	1
6	2408-1096	Centerfodplade aluminium	Central footplate, aluminium	1
7	2408-1093	Skydeskinne-lægpude	Sliding rail - calf rest	2
8	2406-1161	Fodmåtte for fodplade voksen/barn	Footmat for footplate adult/child	1
9	2404-1189	Actuator, ud/ind, central fodplade. 1:51	Actuator, out/in, central footrest. 1:51	1

MC 1122 Quantum

Fodplade centerdel
Footrest center console

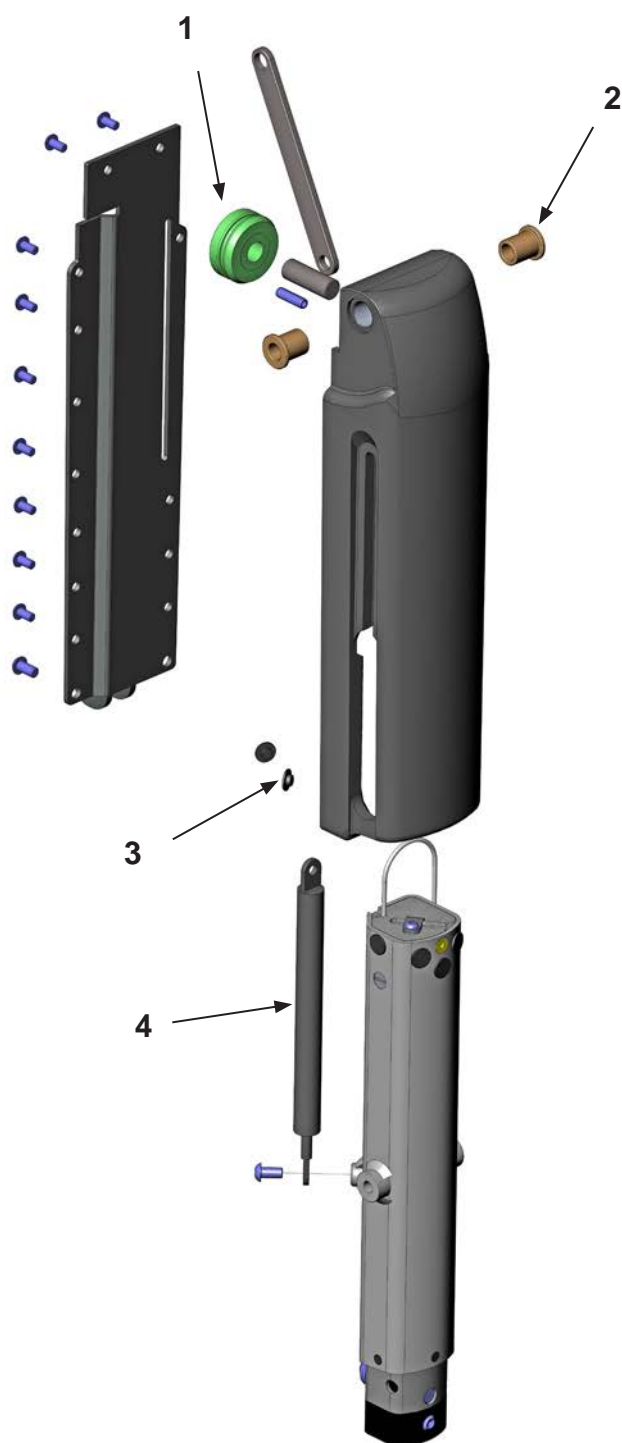

MC 1122 Quantum


Fodplade centerdel

Footrest center console

Pos	Del/part no.	Beskrivelse	Part name	Antal/ quant.
1	8035-1043	Trisse for Ø2,5 wire	Pulley for 2,5 mm metalwire	1
2	2408-1262	10/15x20-20x2,5 Bronze bøsning.	10/15x20-20x2,5 Bronze fitting.	1
3	2408-1097	Afstandsprop	Spacing plug	1
4	2403-1208	Gasfjeder	Suspension damper, 1122	1

MC 1122 Quantum

Stolpe inderdel
Console innerpart

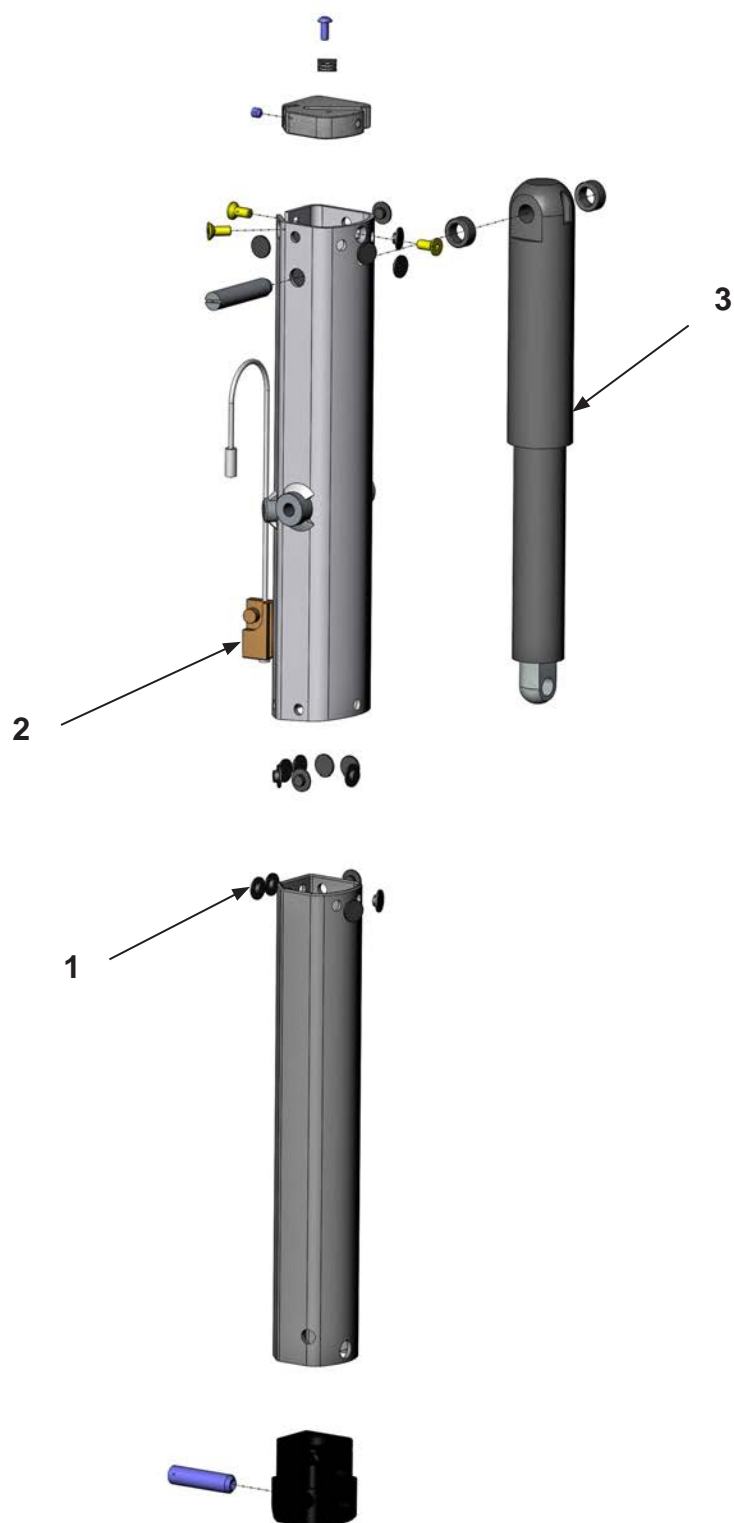


MC 1122 Quantum

Stolpe inderdel
Console innerpart

Pos	Del/part no.	Beskrivelse	Part name	Antal/ quant.
1	2408-1097	Afstandsprop	Spacing plug	1
2	2408-1202	Wire til op/ned	Metalwire for up and down	1
3	2404-1232	Aktuator central fodplade 1:27 med spiral ledning	Aktuator central footrest 1:27 with a spiral cord.	1

MC 1122 Quantum

Centralfodplade, barn, EL.
Central footplate, child, electric

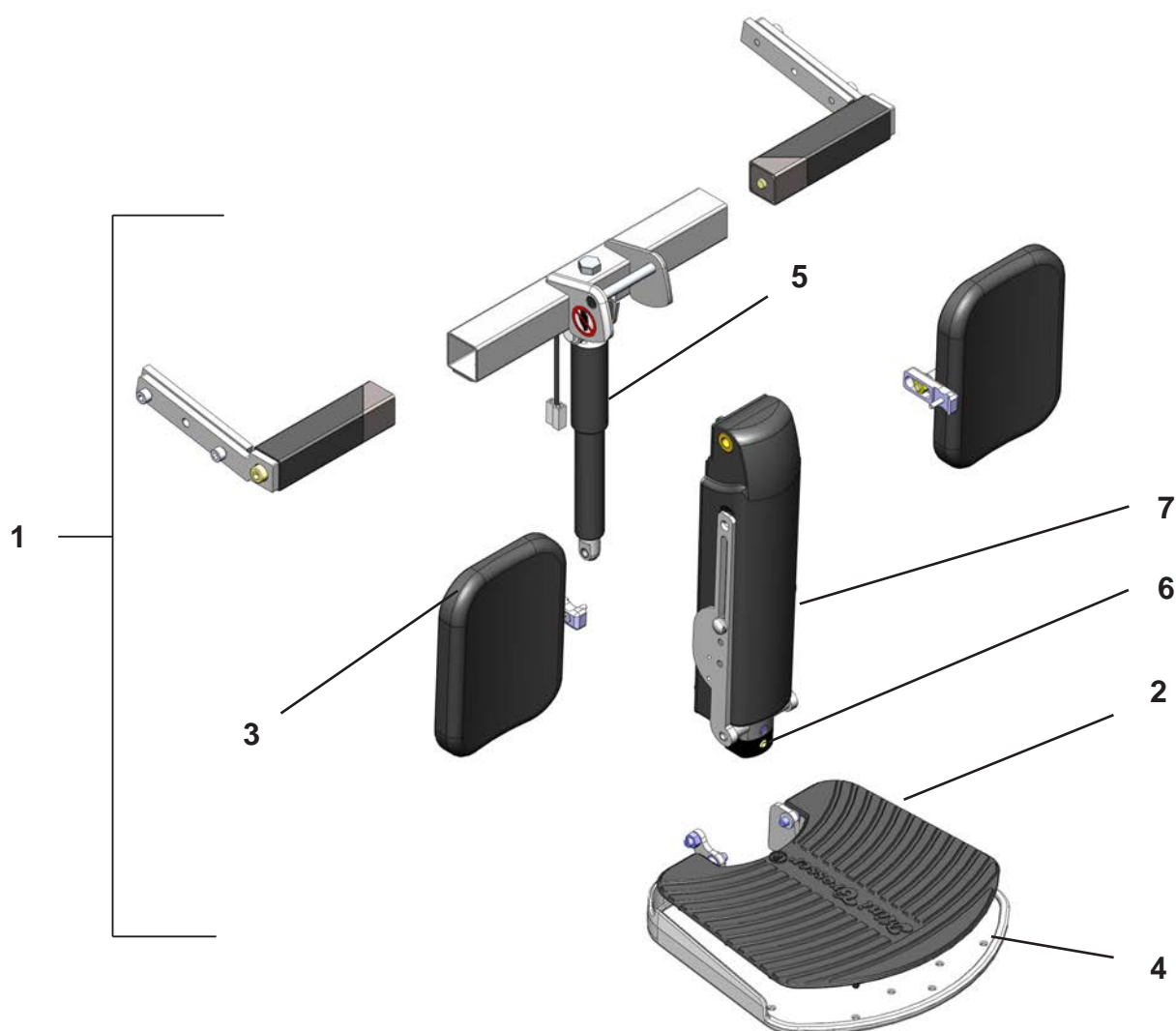


MC 1122 Quantum

Centralfodplade, barn, EL.
 Central footplate, child, electric

Pos	Del/part no.	Beskrivelse	Part name	Antal/ quant.
1	2406-1160	Fodbræt t/ barn (ud-ind/op-ned)	Footplate for child (out-in/up-down)	1
2	2406-1161	Fodmåtte for fodplade voksen/barn	Footmat for footplate adult/child	1
3	2413-1092	Lægpude komplet, barn	Calf rest complete, child	2
4	2408-1096	Centerfodplade aluminium	Central footplate aluminium	1
5	2404-1229	Actuator, ud/ind, central fodplade. 1:51	Actuator in/out, central foodrest. 1:51	1
6	2404-1232	Aktuator op/ned central fodplade 1:27	Actuator up/down, central foodrest 1:27	1
7	1507-1336	Stolpe inderdel	Post, inner part	1

MC 1122 Quantum

Centralfodplade, voksen manuel
Central footplate adult, manual

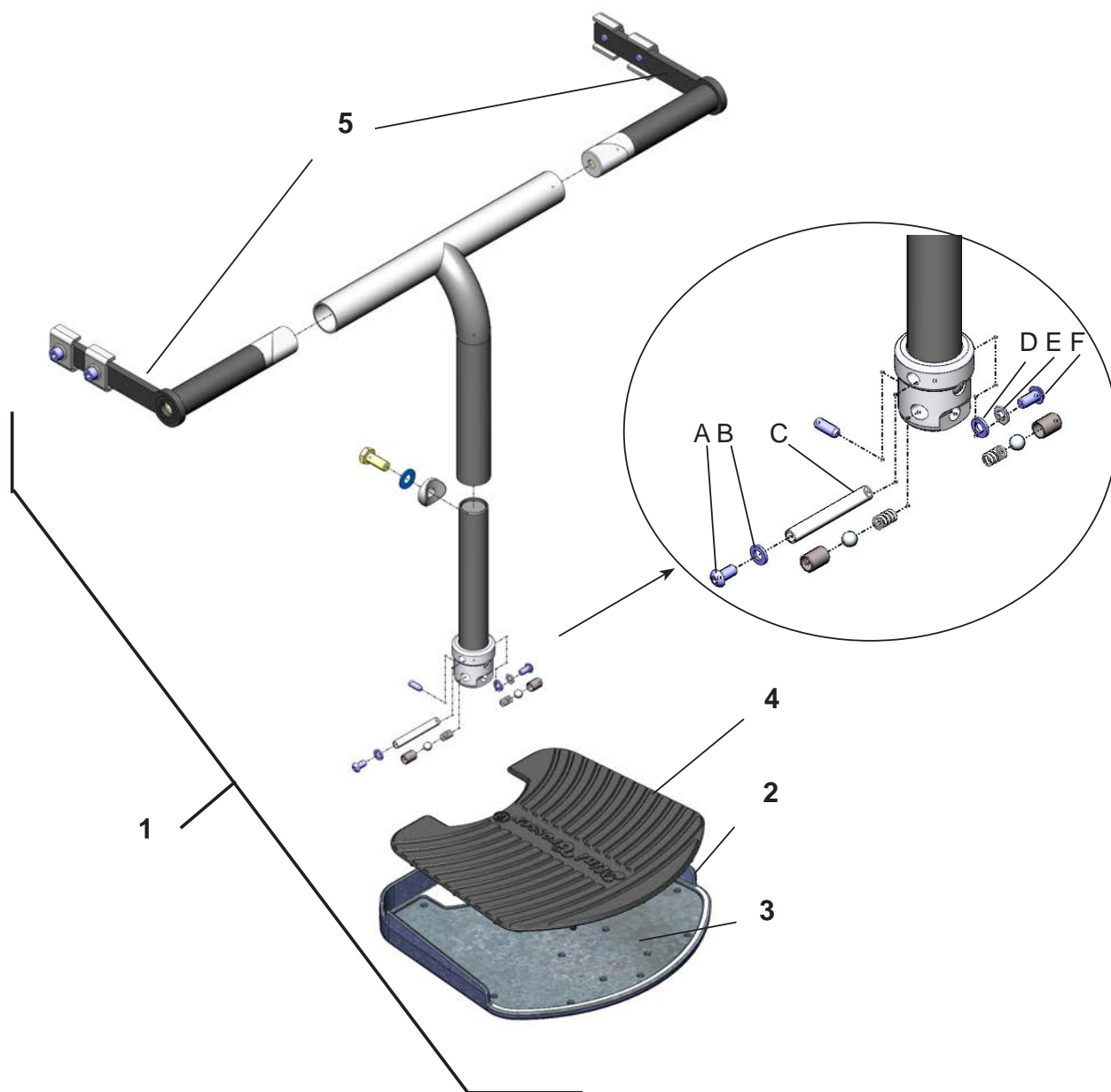


MC 1122 Quantum

Centralfodplade, voksen manuel
 Central footplate adult, manual

Pos	Del/part no.	Beskrivelse	Part name	Antal/ quant.
1	2408-1116	Fodbræt, manuel indstillelig, voksen	Footrest, manual, adult	1
2	2406-1162	Fodplade inkl. måtte samt pos A-F	Footplate incl. mat and pos A-F	1
3	2408-1096	Centerfodplade aluminium	Central footplate aluminium	1
4	2406-1161	Måtte til Centerfodplade aluminium	Mat for Central footplate aluminium	1
5	2408-1236	Torsionsled rund	Torque joint round	2

MC 1122 Quantum

Centralfodplade, Junior manuel
Central footplate, Junior manual

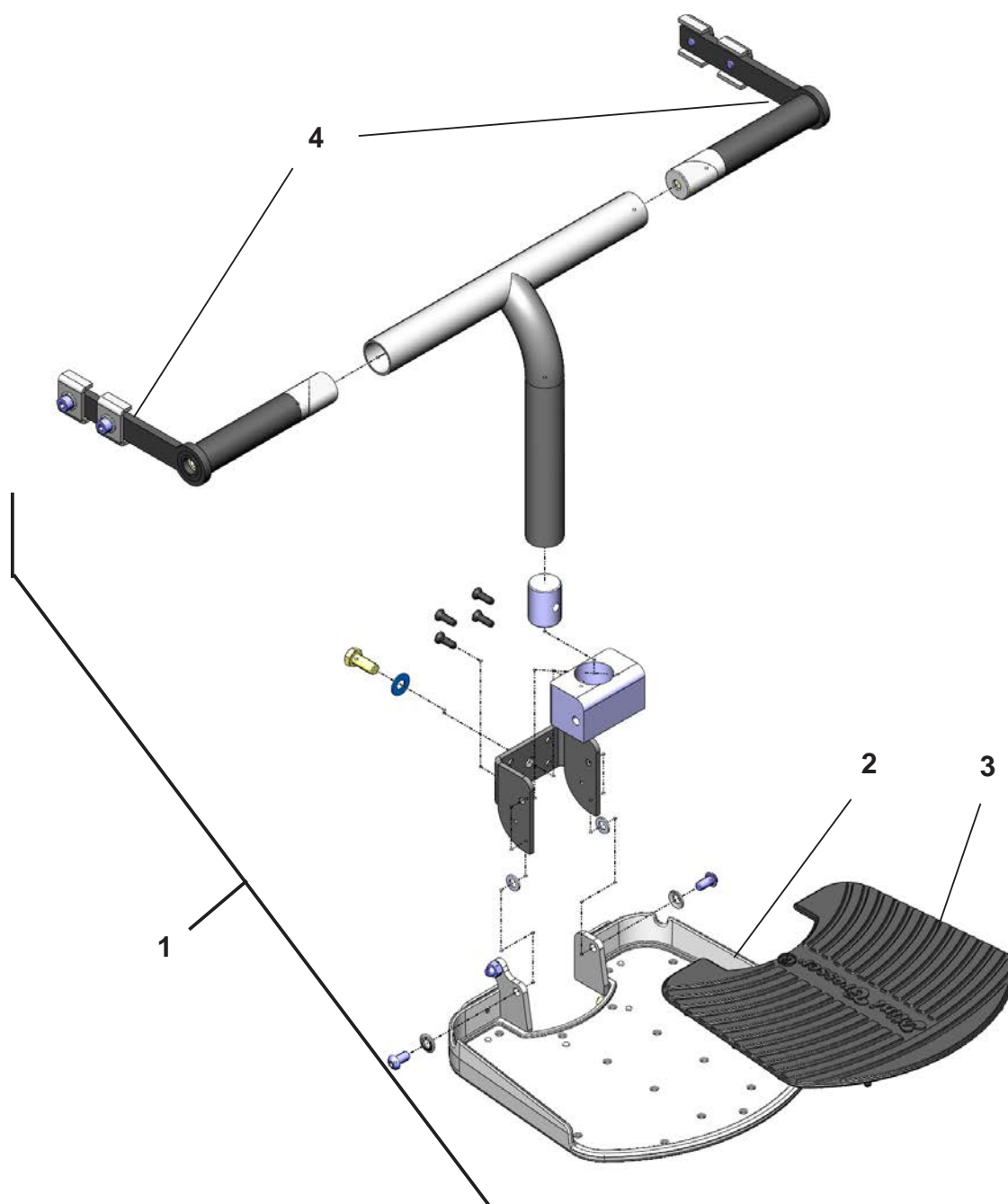

MC 1122 Quantum

Centralfodplade, Junior manuel
Central footplate, Junior manual


Pos	Del/part no.	Beskrivelse	Part name	Antal/ quant.
1	2408-1117	Fodbræt, manuel indstillelig, junior	Footrest, manual, junior	1
2	2413-1113	Centerfodplade aluminium	Central footplate aluminium	1
3	2406-1161	Måtte til Centerfodplade aluminium	Mat for Central footplate aluminium	1
4	2408-1236	Torsionsled rund	Torque joint round	2



MC 1122 Quantum (Electric central footrest - with split footrests)

Artikuler fodplade - komplet øvre samling

Articulating Foot Platform - Complete Upper Assembly

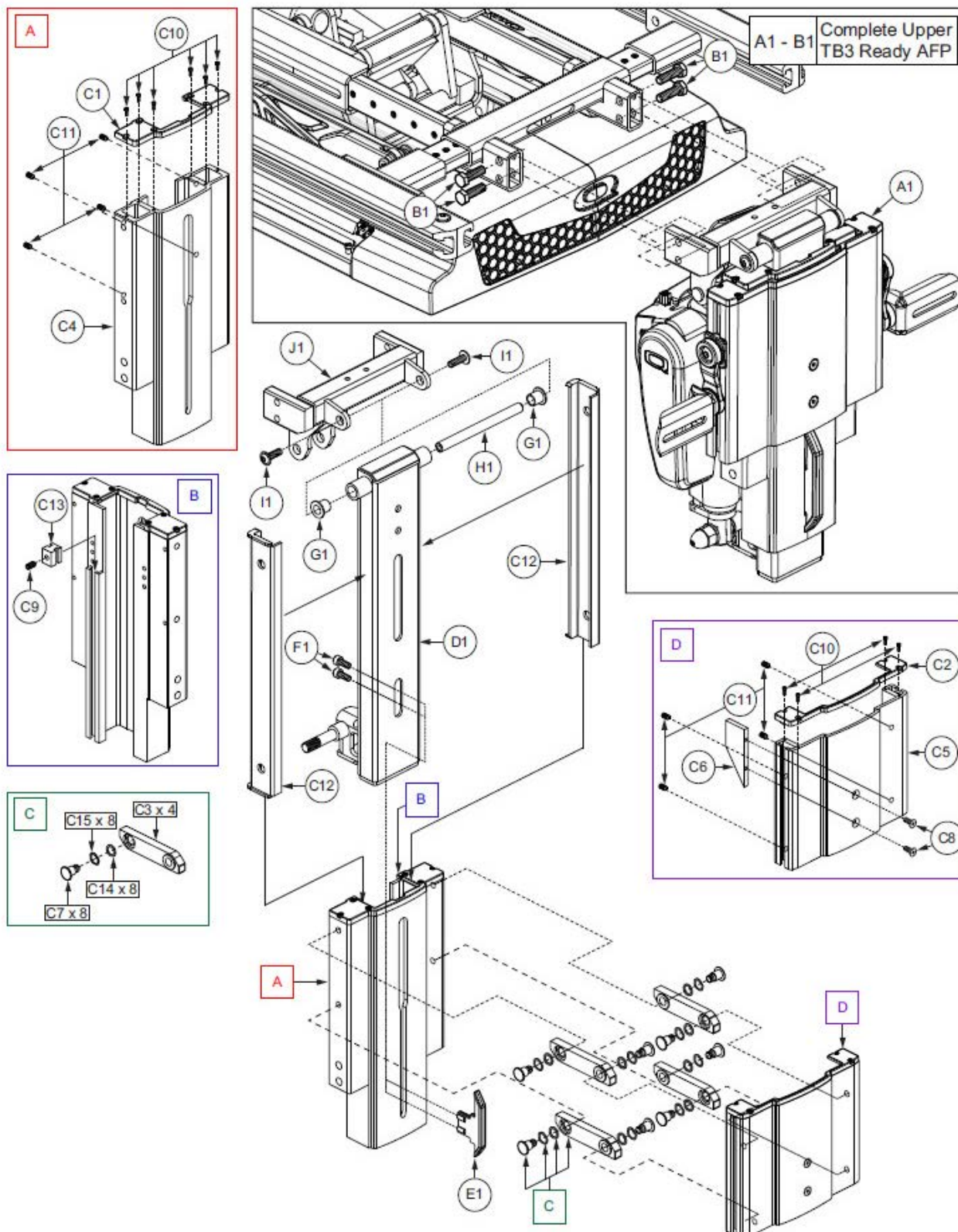




MC 1122 Quantum (Electric central footrest - with split footrests)

Artikulær fodplade - komplet øvre samling
 Articulating Foot Platform - Complete Upper Assembly

REF #	PART #	DESCRIPTION	QTY	MSRP (Euro) ¹¹¹
A1 - B1	RIG131108	LEGREST,ASSY,CENTER MOUNT,ARTICULATING FOOT PLATFORM,W MOUNTING HARDWARE, TRU-BALANCE 3	1	2547.45
A1	RIG131138	LEGREST,ASSY,CENTER MOUNT,ARTICULATING FOOT PLATFORM,W/HARNESSES, TRU-BALANCE 3	1	2536.23
B1	SCRHHCS1412	SCREW,HHCS,BLACK ZINC,M8-1.25 X 16MM,GRADE 10.9,W/NYLON PATCH	4	2.81
C1 - C15	FRMASMB17963	SLIDE,ASSY,CALF PAD,CENTER MOUNT ARTICULATING FOOT PLATFORM,E-1404-005	1	348.35
C1	DWR1404E003	EXTRUSION END CAP,ARTICULATING FOOT PLATFORM,E-1404-003	1	N/A
C2	DWR1404E004	EXTRUSION END CAP,OUTER,ARTICULATING FOOT PLATFORM,E-1404-004	1	N/A
C3	FRMASMB17965	MOUNTING BRACKET,ASSY,CALF PAD LINK, TRU-BALANCE POWER ARTICULATING FOOT PLATFORM,C-1404-037	4	N/A
C4	DWR1404D021	SLIDE EXTRUSION,MAIN, TRU-BALANCE POWER ARTICULATING FOOT PLATFORM,D-1404-021	1	118.08
C5	DWR1404D022	SLIDE EXTRUSION,CALF PAD,CENTER MOUNT ARTICULATING FOOT PLATFORM,D-1404-022	1	N/A
C6	DWR1404C012	BRACKET,CALF PAD UPPER RAMP, TRU-BALANCE POWER ARTICULATING FOOT PLATFORM,C-1404-012	1	N/A
C7	DWR1404C034	PIN,PIVOT,CALF PAD LINK, TRU-BALANCE POWER ARTICULATING FOOT PLATFORM,C-1404-034	8	N/A
C8	HDW120030	SCREW,SHFH,BLACK ZINC,M5-.8 X 12MM,GRADE 10.9,NYLON PATCH	2	N/A
C9	SCRSETS1105	SCREW,SET,M6-1.0T X 8MM,CUP POINT,W/NYLON PATCH	1	2.81
C10	SCRSHCS1355	SCREW,SHCS,M2.5-0.45 X 8MM,W/NYLON PATCH,C-1005-100	10	2.81
C11	SCRSETS1151	SCREW,SET,BLACK ZINC,M5-.8 X 8MM,W/NYLON PATCH,CONE POINT	8	2.86
C12	DWR1404D019	BUSHING,11.22"L,ADJUSTABLE CENTER MOUNT FOOT PLATFORM,D-1404-019	2	4.72
C13	DWR1404C033	NUT,BLACK,M6 X 1.0,ARTICULATING LIMIT SWITCH,C-1404-033	1	N/A
C14	WASSTND1214	WASHER,FLAT,BLACK ACETAL,8.0MM X 13MM X 0.50MM THK	8	2.81
C15	WASWAVY1030	WASHER,WAVE DISC SPRING,0.350 ID X 0.492 OD X 0.007 THK,302 SS,(MCMMASTER:9714K280)	8	2.81
D1	DWR1404D052	SLIDE TUBE WELDMENT,MAIN,CENTER MOUNT ARTICULATING FOOT PLATFORM,D-1404-052	1	N/A
E1	DWR1404C028	BRACKET,CALF PAD LOWER RAMP,PLASTIC, TRU-BALANCE POWER ARTICULATING FOOT PLATFORM,C-1404-028	1	2.81
F1	SCR8110014	SCREW,SHCS,BLACK ZINC,M5-.8 X 8MM,ASTM A574M,NYLON PATCH	2	2.81
G1	BRGFLNG1065	BEARING,FLANGE,0.375" ID X 0.46875" OD X 0.500" TH,0.687" FOD X 0.046" FTH,(IGUS: TFI-0607-08)	2	23.19
H1	DWR1404C063	PIVOT ROD,UPPER, TRU-BALANCE POWER ARTICULATING FOOT PLATFORM,C-1404-063	1	10.12
I1	SCRASBH1331	SCREW,BHSCS,BLACK ZINC,M6-1.0 X 20MM,FLANGED,GRADE 10.9,W/NYLON PATCH,C-1005-096	2	2.81
J1	MEC134031	UPPER PIVOT WELDMENT,CENTERMOUNT ARTICULATING FOOT PLATFORM,TRU-BALANCE 3	1	31.21

MC 1122 Quantum (Electric central footrest - with split footrests)

Artikulær fodplade - Aktuatorer og afdækning
 Articulating Foot Platform - Actuators and Shrouds

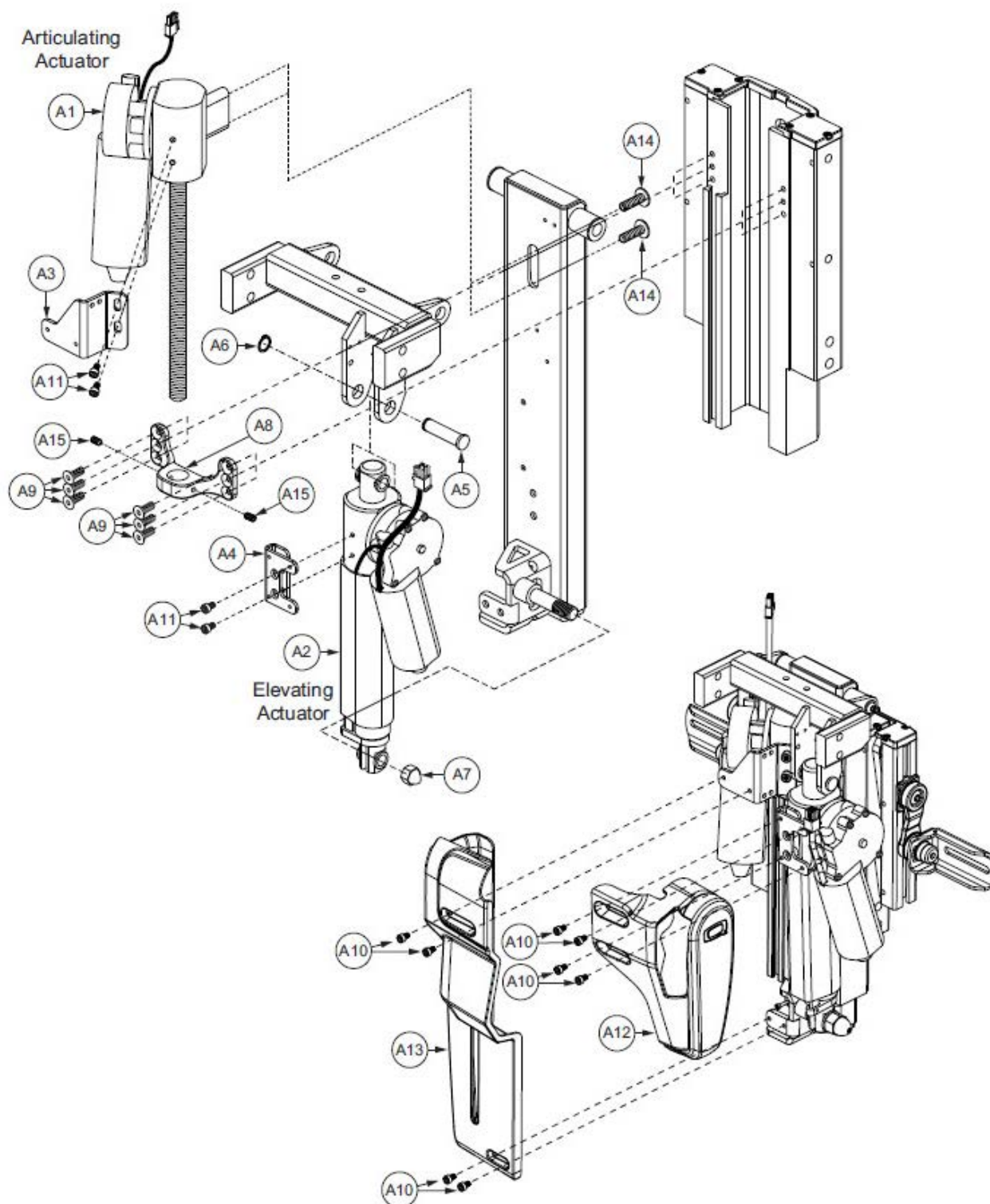




MC 1122 Quantum (Electric central footrest - with split footrests)

Artikulær fodplade - Aktuatorer og afdækning
Articulating Foot Platform - Actuators and Shrouds

REF #	PART #	DESCRIPTION	QTY	MSRP (Euro) ¹¹
A1	ACCACTR1062	ACTUATOR, ARTICULATING, TRU-BALANCE POWER ARTICULATING FOOT PLATFORM, H-1404-001, (MBL: LL-3001)	1	618.15
A2	ACT8110005	ACTUATOR, ELEVATE, 96MM STROKE, TRU-BALANCE POWER ARTICULATING FOOT PLATFORM, (MBL: LL-3001)	1	708.47
A3	FRM8110135	BRACKET, UPPER, LH COVER	1	N/A
A4	FRM8110136	BRACKET, UPPER, RH COVER	1	N/A
A5	DWR1404B010	CLEVIS PIN, ACTUATOR, TRU-BALANCE POWER ARTICULATING FOOT PLATFORM, UPPER, B-1404-010	1	15.93
A6	RNGRTAN1035	RETAINING RING, BLACK ZINC, 0.375", EXTERNAL, B-1005-084	1	6.81
A7	NUTACRN1036	NUT, ACORN, HIGH CROWN, BLIND, BLACK ZINC, M10-1.50, W/NYLON PATCH	1	N/A
A8	DWR1404C024	BRACKET, ARTICULATING MOTOR MOUNT, TRU-BALANCE POWER ARTICULATING FOOT PLATFORM, C-1404-024	1	N/A
A9	SCRSHFH1015	SCREW, SHFH, BLACK ZINC, M5-.8 X 12MM, GRADE 10.9	6	2.81
A10	SCR8110015	SCREW, SHCS, BLACK ZINC, M4-.7 X 8MM, ASTM A574M, NYLON PATCH	8	3.16
A11	SCR8110014	SCREW, SHCS, BLACK ZINC, M5-.8 X 8MM, ASTM A574M, NYLON PATCH	4	2.81
A12	PLS8110044	COVER, PLASTIC, ELEVATING MOTOR, CENTER MOUNT ARTICULATING FOOT PLATFORM	1	6.75
A13	PLS8110043	COVER, PLASTIC, ARTICULATING MOTOR, CENTER MOUNT ARTICULATING FOOT PLATFORM	1	6.75
A14	SCRASBH1118	SCREW, BHSCS, BLACK ZINC CHROMATE, M6-1.0 X 12MM	2	2.81
A15	SCRSETS1149	SCREW, SET, M5-0.8 X 5MM, FLAT POINT, W/NYLON PATCH, C-1005-101	2	2.81

MC 1122 Quantum (Electric central footrest - with split footrests)

Artikulær fodplade - Elektronisk
 Articulating Foot Platform - Electronics

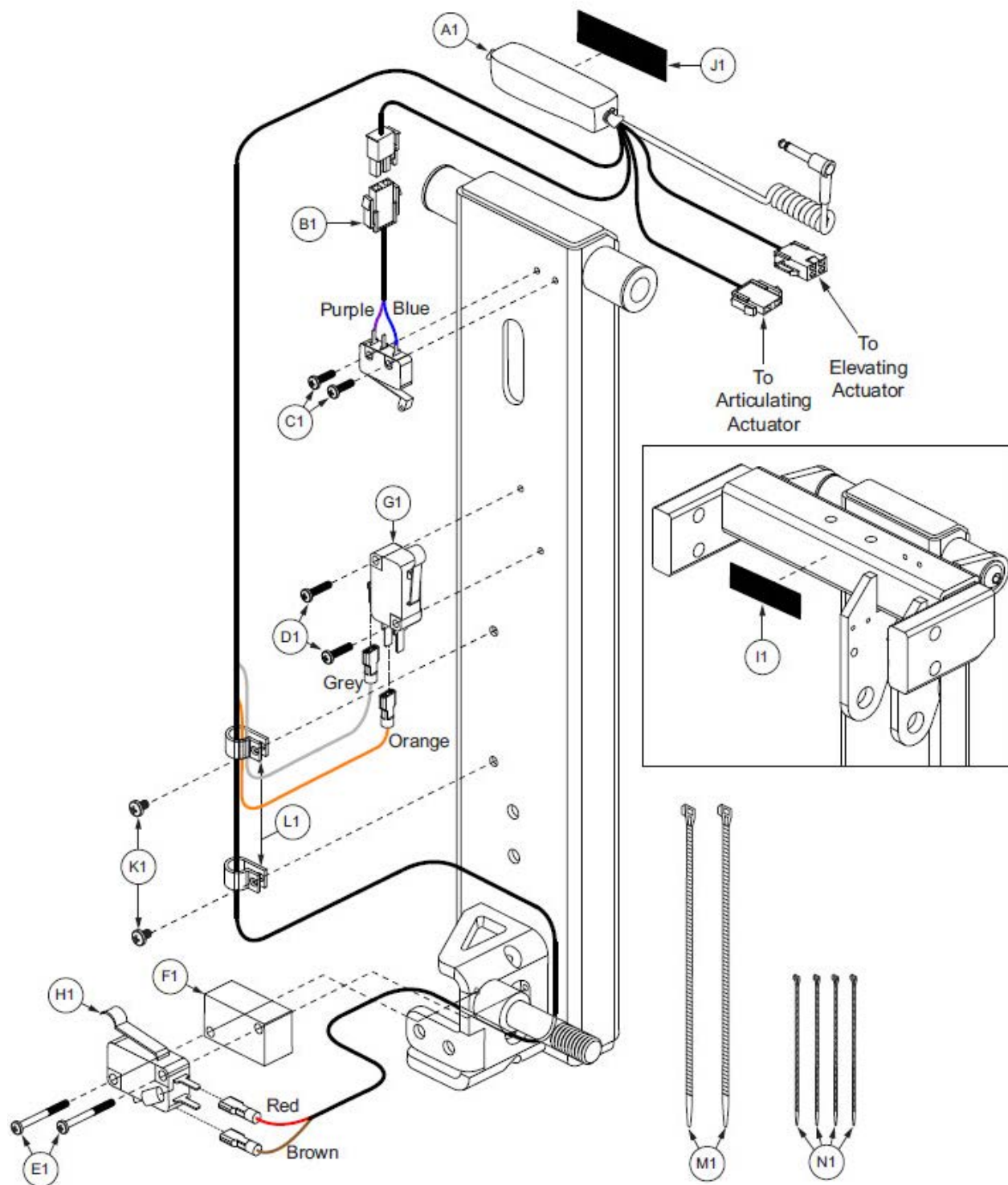

MC 1122 Quantum (Electric central footrest - with split footrests)


Arlikulær fodplade - Elektronisk
 Articulating Foot Platform - Electronics

REF #	PART #	DESCRIPTION	QTY	MSRP (Euro) ¹¹
A1	ELEASMB7110406	ELECTRONIC,ASSY,PCB,CONTROL HARNESS,W/MONO PLUG,ARTICULATING FOOT PLATFORM	1	97.97
B1	HAR8110014	ELECTRONIC,HARNESS,ARTICULATING ACTUATOR,REAR STOP SWITCH,ARTICULATING FOOT PLATFORM	1	39.91
C1	SCRPPH1454	SCREW,PPH,M2 X 0.4 X 10MM,304 SS	2	2.81
D1	SCRPPH1455	SCREW,PPH,M2.5 X 0.45 X 16MM,304 SS	2	2.81
E1	SCRPPH1456	SCREW,PPH,M2.5 X 0.45 X 30MM,304 SS	2	2.81
F1	DWR1404B002	SPACER,BLACK NYLON,0.63"W X 1.09"L X 0.50"TH,LOWER SWITCH,B-1404-002	1	N/A
G1	SWTMCRO1113	ELECTRONIC,SWITCH,MICRO,15 AMP,W/SIMULATED ROLLER,(OMRON: V-15G4-1C25-K)	1	21.12
H1	SWT8110004	ELECTRONIC,SWITCH,MICRO,15AMP,W/SIMULATED ROLLER,6A DIODE OPPOSITE NUMBERS,CATHODE TO NC,(OMRON:V-15G4-1C25-K)	1	21.77
I1	VELLOOP1007	REUSABLE FASTENER,LOOP,2" X 1",W/ADHESIVE BACK	1	2.81
J1	VELHOOK1054	FASTENER,1"W X 2"L,REUSABLE,HOOK	1	2.81
K1	SCR8110014	SCREW,SHCS,BLACK ZINC,M5-.8 X 8MM,ASTM A574M,NYLON PATCH	2	2.81
L1	CLP8120001	CLAMP,CABLE,BLACK NYLON,5/8"D,P-CLIP	2	2.86
M1	PLSTIE1004	TIE WRAP,BLACK,8"	2	2.81
N1	PLSTIE1001	TIE WRAP,BLACK,4"	4	2.87

MC 1122 Quantum (Electric central footrest - with split footrests)

Artikuleret fodplade - 10-16" nedre forlænger med fodplade

Articulating Foot Platform - 10-16" Lower Extensions with Footplates

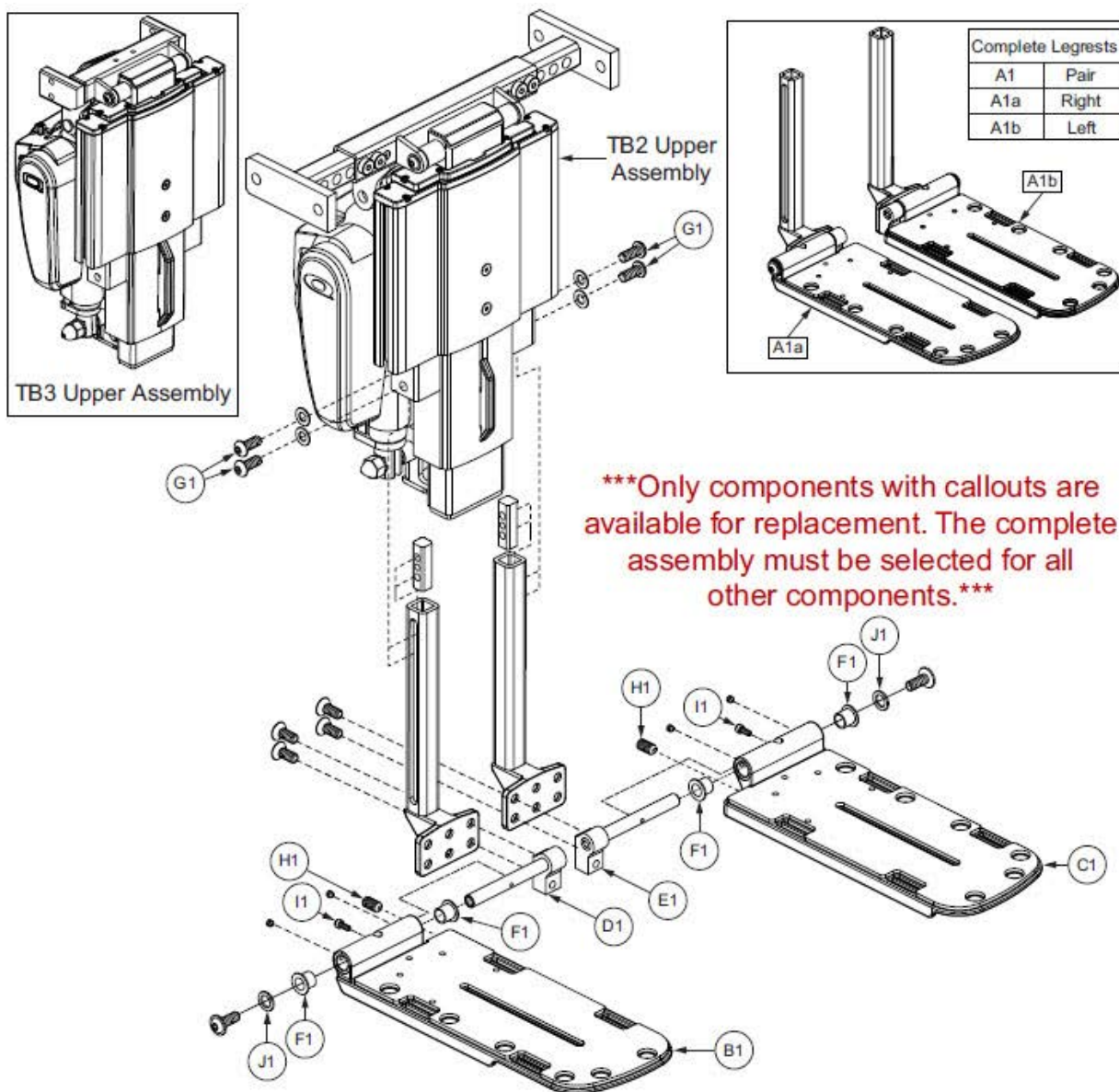

REF #	PART #	DESCRIPTION	QTY	MSRP (Euro) ¹¹ⁱ
A1	RIGASMB7120033	FOOTPLATE, ASSY, 10-16" EXT, CENTER MOUNT AFP, PAIR	1	431.82
A1a	FRMASMB17949	FOOTPLATE, ASSY, 5" X 12", RIGHT	1	205.79
A1b	FRMASMB17948	FOOTPLATE, ASSY, 5" X 12", LEFT	1	205.79
B1	DWR1304D068	FOOTPLATE, 5.00" X 12.00, W/MAT, RIGHT, D-1304-068	1	141.7
C1	DWR1304D069	FOOTPLATE, 5.00" X 12.00", W/MAT, LEFT, D-1304-069	1	141.7
D1	DWR1404C008	PIVOT BRACKET WELDMENT, CENTER MOUNT LEGREST, RIGHT	1	33.74
E1	DWR1404C007	PIVOT BRACKET WELDMENT, CENTER MOUNT LEGREST, LEFT	1	33.74
F1	BRGFLNG1066	BEARING, FLANGE, .4375" ID X .531" OD X .75" FOD X .046" FTH	2	6.76
G1	SCRASBH1251	SCREW, BHSCS, M8 X 1.25T X 20MM, GR 10.9, W/PATCH	2	2.81
H1	SCRSETS1150	SCREW, SET, M10-1.25 X 25MM, CUP POINT, GR 10.9, W/PATCH	1	2.81
I1	SCRSHCS1127	SCREW, SHCS, BLACK ZINC, M4-.7 X 10MM, GRADE 8	1	2.87
J1	WASNYLN1128	WASHER, NYLON, 0.438" ID X 0.675" OD X 0.0625" TH	1	6.43




medemagruppen

MC 1122 Quantum (Electric central footrest - with split footrests)

Artikulær fodplade - 13-19" nedre forlænger med fodplade

Articulating Foot Platform - 13-19" Lower Extensions with Footplates

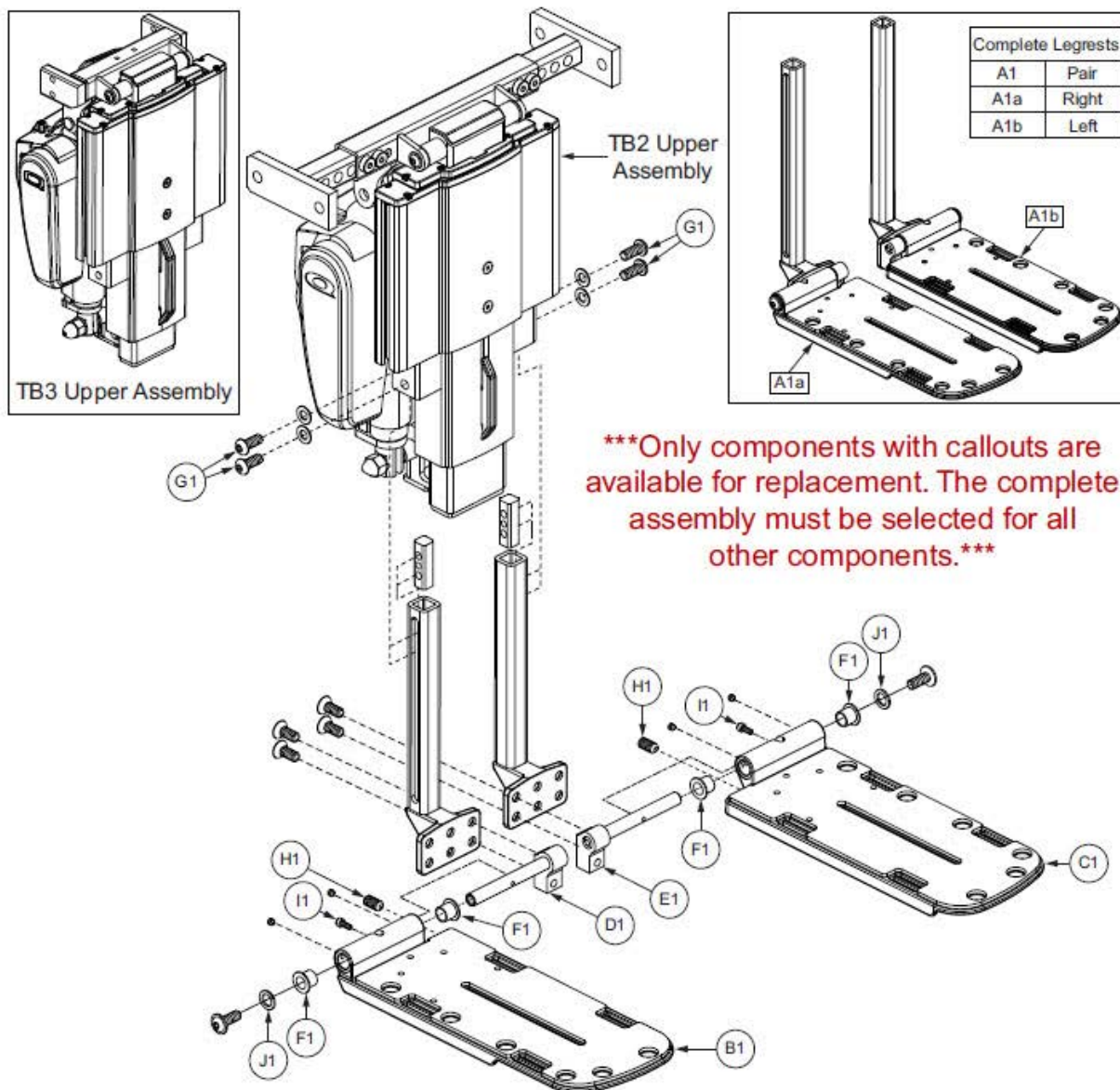


Only components with callouts are available for replacement. The complete assembly must be selected for all other components.

REF #	PART #	DESCRIPTION	QTY	MSRP (Euro) ¹¹¹
A1	RIGASMB7120034	FOOTPLATE, ASSY, 13-19" EXT, CENTER MOUNT AFP, PAIR	1	431.82
A1a	RIGASMB7120004	FOOTPLATE, ASSY, 5" X 12", RIGHT, 13"-19"	1	215.91
A1b	RIGASMB7120005	FOOTPLATE, ASSY, 5" X 12", LEFT, 13"-19"	1	215.91
B1	DWR1304D068	FOOTPLATE, 5.00" X 12.00, W/MAT, RIGHT, D-1304-068	1	141.7
C1	DWR1304D069	FOOTPLATE, 5.00" X 12.00", W/MAT, LEFT, D-1304-069	1	141.7
D1	DWR1404C008	PIVOT BRACKET WELDMENT, CENTER MOUNT LEGREST, RIGHT	1	33.74
E1	DWR1404C007	PIVOT BRACKET WELDMENT, CENTER MOUNT LEGREST, LEFT	1	33.74
F1	BRGFLNG1066	BEARING, FLANGE, .4375" ID X .531" OD X .75" OD X .046" TH	2	6.76
G1	SCRASBH1251	SCREW, BHSCS, M8 X 1.25T X 20MM, GR 10.9, W/PATCH	2	2.81
H1	SCRSETS1150	SCREW, SET, M10-1.25 X 25MM, CUP POINT, GR 10.9, W/PATCH	1	2.81
I1	SCRSHCS1127	SCREW, SHCS, BLACK ZINC, M4-7 X 10MM, GRADE 8	1	2.87
J1	WASNLYN1128	WASHER, NYLON, 0.438" ID X 0.675" OD X 0.0625" TH	1	6.43

MC 1122 Quantum (Electric central footrest - with split footrests)

Centermonteret fodplade - komplet samlet
 Center Mount Foot Platform - Complete Assemblies

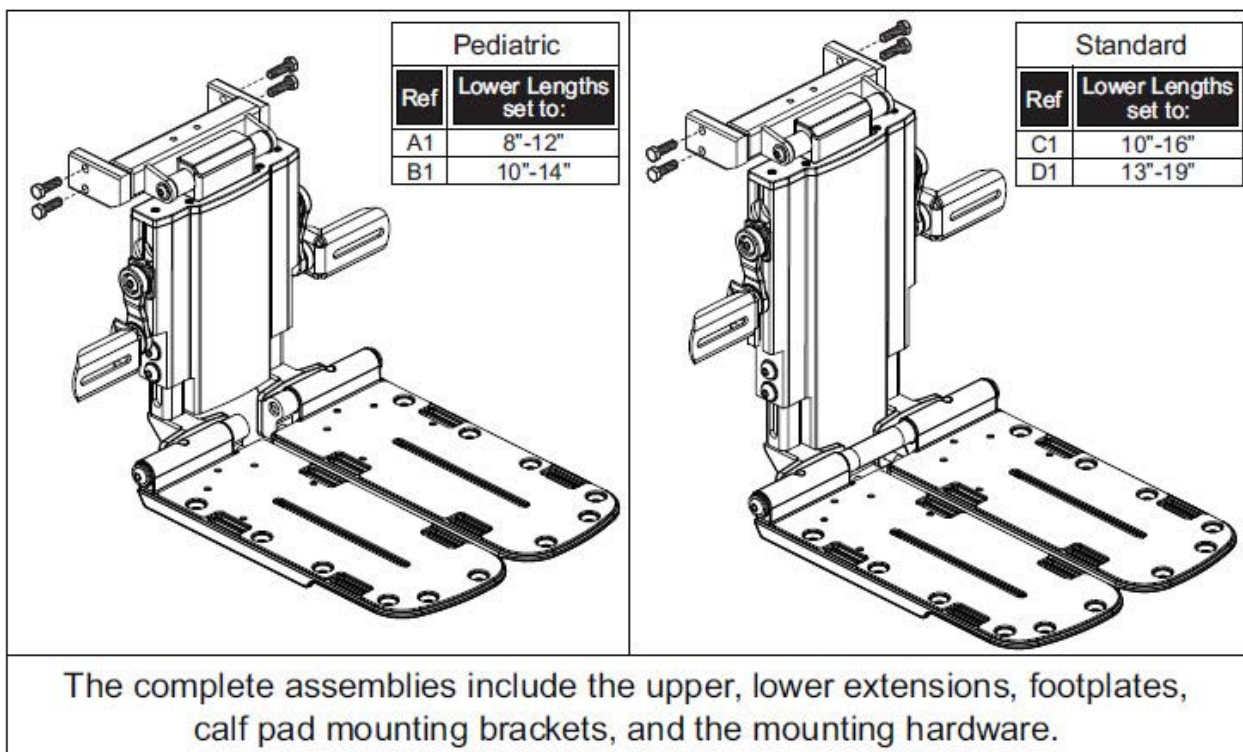


REF #	PART #	DESCRIPTION	QTY	MSRP (Euro) ¹¹
A1	RIG131111	ADJUSTABLE CENTER MOUNT FOOT PLATFORM,ASSY,8"-12"L,PEDIATRIC,W/MOUNTING HARDWARE	1	N/A
B1	RIG131111	ADJUSTABLE CENTER MOUNT FOOT PLATFORM,ASSY,8"-12"L,PEDIATRIC,W/MOUNTING HARDWARE	1	N/A
C1	RIG131109	ADJUSTABLE CENTER MOUNT FOOT PLATFORM,ASSY,10"-16"L,W/MOUNTING HARDWARE	1	N/A
D1	RIG131110	ADJUSTABLE CENTER MOUNT FOOT PLATFORM,ASSY,13"-19"L,W/MOUNTING HARDWARE	1	N/A

MC 1122 Quantum (Electric central footrest - with split footrests)

AFP/Centermonteret fodplade - lægstøtter

AFP/Center Mount Foot Platform - Calf Pads

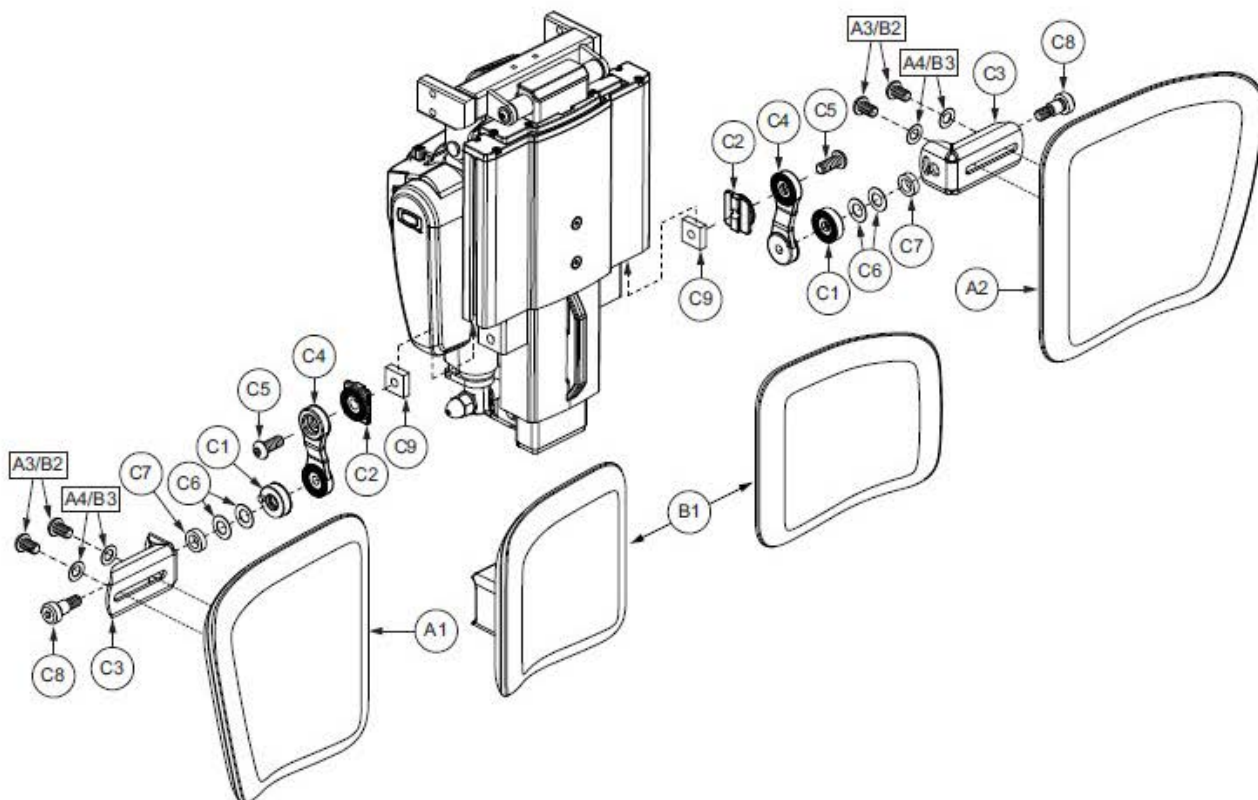


REF #	PART #	DESCRIPTION	QTY	MSRP (Euro) ¹¹ⁱ
A1 - A4	RIGASMB7120019	CALF PAD,ASSY,LARGE,CENTER MOUNT FOOT PLATFORM	1	109.29
A1	DWR1404D031	CALF PAD,8" X 8",POWER AFP,RIGHT	1	46.22
A2	DWR1404D030	CALF PAD,8" X 8",POWER AFP,LEFT	1	46.22
A3	SCRSHCS1095	SCREW,SHCS,BLACK ZINC,M6-1.0 X 12MM,GR 10.9,W/PATCH	4	2.81
A4	WASSTND1196	WASHER,FLAT,BLACK ZINC,0.25" ID X 0.50" OD X 0.03125" TH	4	2.81
B1 - B3	RIGASMB7120018	CALF PAD,ASSY,SMALL,CENTER MOUNT FOOT PLATFORM	1	103.68
B1	DWR1404D029	CALF PAD,6" X 6",POWER AFP	2	50.62
B2	SCRSHCS1095	SCREW,SHCS,BLACK ZINC,M6-1.0 X 12MM,GR 10.9,W/PATCH	4	2.81
B3	WASSTND1196	WASHER,FLAT,BLACK ZINC,0.25" ID X 0.50" OD X 0.03125" TH	4	2.81
C1 - C9	FRMASMB17964	PIVOT BRACKET,ASSY,CALF PAD SUPPORT,POWER AFP	2	44.27
C1	DWR1404C031	STANDOFF,CALF SUPPORT,OUTSIDE,CENTER MOUNT,C-1404-031	1	3.37
C2	DWR1404C032	STANDOFF,CALF SUPPORT,INSIDE,CENTER MOUNT,C-1404-032	1	6.07
C3	DWR1404D005	BRACKET,CALF SUPPORT,CENTER MOUNT LEGREST,D-1404-005	1	9.44
C4	DWR1404D016	BRACKET,CALF SUPPORT STANDOFF,CENTER MOUNT,D-1404-016	1	8.11
C5	SCRASBH1251	SCREW,BHSCS,M8 X 1.25 X 20MM,GR 10.9,W/PATCH	1	2.81
C6	WASBELV1011	WASHER,BELLEVILLE,10.2MM ID X 18.8MM OD X 0.35MM TH	2	3.03
C7	WAS8110007	WASHER,FLAT,BLACK NYLON,10.5MM ID X 16MM OD X 5MM THK	1	2.81
C8	BLT8110009	SHOULDER BOLT, M10 X 12MM, CLASS 12.9	1	2.81
C9	DWR1404B001	NUT,BLACK ZINC,M8-1.25,B-1404-001	1	2.81

MC 1122 Quantum (Electric central footrest - with split footrests)

Centermonteret fodlade - Pediatric

Center Mount Foot Platform - Pediatric

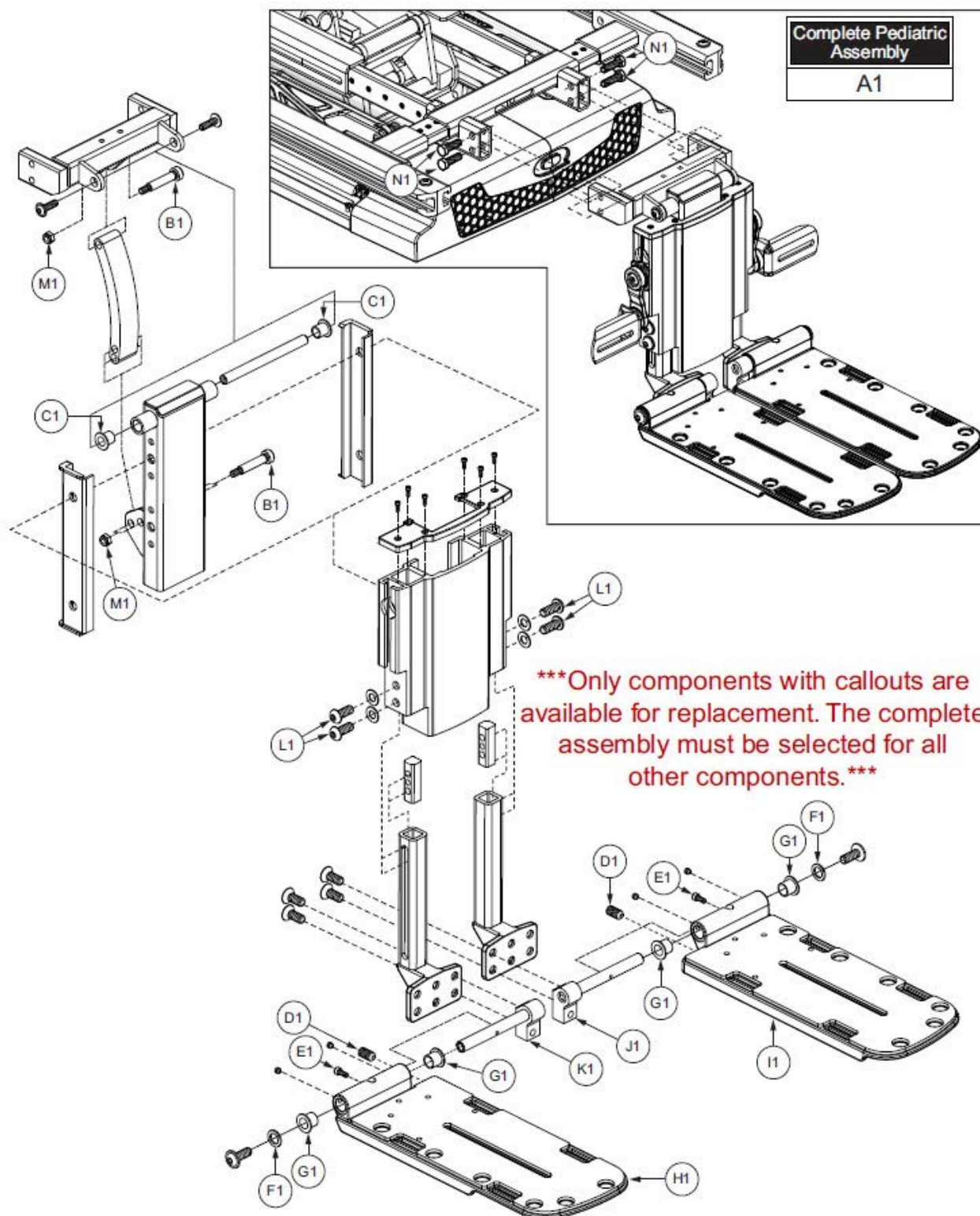



MC 1122 Quantum (Electric central footrest - with split footrests)

Centermonteret fodlade - Pediatric


Center Mount Foot Platform - Pediatric

REF #	PART #	DESCRIPTION	QTY	MSRP (Euro) ¹¹ⁱ
A1	RIG130092	ADJUSTABLE CENTER MOUNT FOOT PLATFORM,ASSY,PEDIATRIC,TRU-BALANCE 3	1	841.85
B1	BLTASHL1131	BOLT,SHOULDER,BLACK ZINC,8MM X 35MM	2	3
C1	BRGFLNG1065	BEARING,FLANGE,0.375" ID X 0.46875" OD X 0.500" TH,0.687" FOD X 0.046" FTH,(IGUS: TFI-0607-08)	2	23.19
D1	SCRSETS1150	SCREW,SET,BLACK ZINC,M10-1.25 X 25MM,CUP POINT,GRADE 10.9,W/NYLON PATCH,C-1005-102	2	2.81
E1	SCRSHCS1127	SCREW,SHCS,BLACK ZINC,M4-.7 X 10MM,GRADE 8	2	2.87
F1	WASNLYN1128	WASHER,NYLON,0.438" ID X 0.675" OD X 0.0625" TH	2	6.43
G1	BRGFLNG1066	BEARING,FLANGE,0.4375" ID X 0.531" OD X 0.75" FOD X 0.046" FTH,B-1005-110,(IGUS: GFI-0708-08)	4	6.76
H1	DWR1304D068	FOOTPLATE,5.00" X 12.00,W/MAT,RIGHT,D-1304-068	1	141.7
I1	DWR1304D069	FOOTPLATE,5.00" X 12.00",W/MAT,LEFT,D-1304-069	1	141.7
J1	DWR1404C007	PIVOT BRACKET WELDMENT,BLACK,CENTER MOUNT LEGREST,LEFT,C-1404-007	1	33.74
K1	DWR1404C008	PIVOT BRACKET WELDMENT,BLACK,CENTER MOUNT LEGREST,RIGHT,C-1404-008	1	33.74
L1	SCRASBH1251	SCREW,BHSCS,BLACK ZINC,M8 X 1.25T X 20MM,GRADE 10.9,W/NYLON PATCH	4	2.81
M1	NUTNYLK1019	NUT,NYLOCK,BLACK ZINC,M6-1.0	2	2.87
N1	SCRHHCS1412	SCREW,HHCS,BLACK ZINC,M8-1.25 X 16MM,GRADE 10.9,W/NYLON PATCH	4	2.81



MC 1122 Quantum (Electric central footrest - with split footrests)

Centermonteret fodplade - Standard
Center Mount Foot Platform - Standard

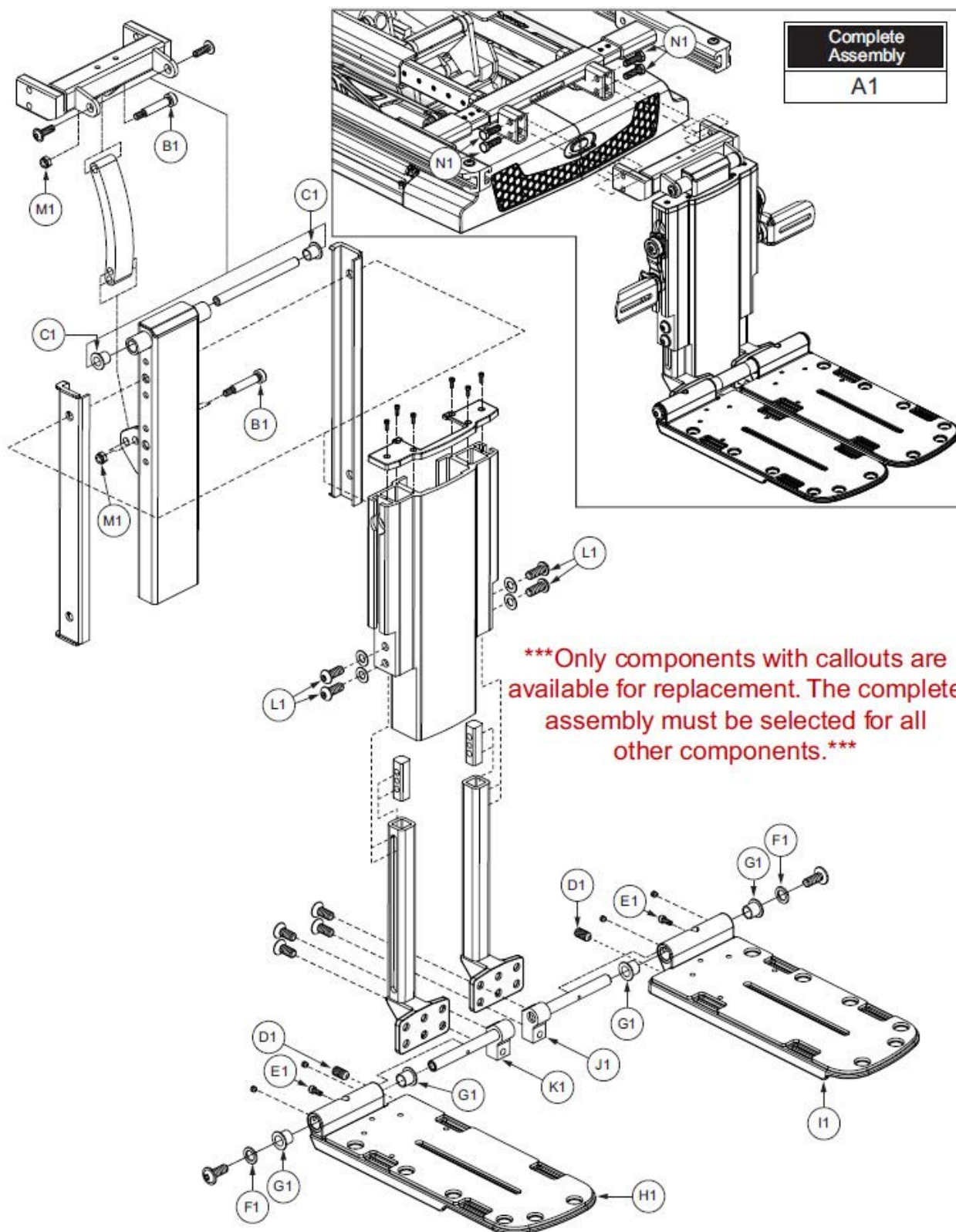

MC 1122 Quantum (Electric central footrest - with split footrests)


Centermonteret fodplade - Standard

Center Mount Foot Platform - Standard

REF #	PART #	DESCRIPTION	QTY	MSRP (Euro) ¹¹
A1	RIG130091	ADJUSTABLE CENTER MOUNT FOOT PLATFORM,ASSY,TRU-BALANCE 3	1	841.85
B1	BLTASHL1131	BOLT,SHOULDER,BLACK ZINC,8MM X 35MM	2	3
C1	BRGFLNG1065	BEARING,FLANGE,0.375" ID X 0.46875" OD X 0.500" TH,0.687" FOD X 0.046" FTH,(IGUS: TFI-0607-08)	2	23.19
D1	SCRSETS1150	SCREW,SET,BLACK ZINC,M10-1.25 X 25MM,CUP POINT,GRADE 10.9,W/NYLON PATCH,C-1005-102	2	2.81
E1	SCRSHCS1127	SCREW,SHCS,BLACK ZINC,M4- 7 X 10MM,GRADE 8	2	2.87
F1	WASNLYN1128	WASHER,NYLON,0.438" ID X 0.675" OD X 0.0625" TH	2	6.43
G1	BRGFLNG1066	BEARING,FLANGE,0.4375" ID X 0.531" OD X 0.75" FOD X 0.046" FTH,B-1005-110,(IGUS: GFI-0708-08)	4	6.76
H1	DWR1304D068	FOOTPLATE,5.00" X 12.00,W/MAT,RIGHT,D-1304-068	1	141.7
I1	DWR1304D069	FOOTPLATE,5.00" X 12.00",W/MAT,LEFT,D-1304-069	1	141.7
J1	DWR1404C007	PIVOT BRACKET WELDMENT,BLACK,CENTER MOUNT LEGREST,LEFT,C-1404-007	1	33.74
K1	DWR1404C008	PIVOT BRACKET WELDMENT,BLACK,CENTER MOUNT LEGREST,RIGHT,C-1404-008	1	33.74
L1	SCRASBH1251	SCREW,BHSCS,BLACK ZINC,M8 X 1.25T X 20MM,GRADE 10.9,W/NYLON PATCH	4	2.81
M1	NUTNYLK1019	NUT,NYLOCK,BLACK ZINC,M6-1.0	2	2.87
N1	SCRHHCS1412	SCREW,HHCS,BLACK ZINC,M8-1.25 X 16MM,GRADE 10.9,W/NYLON PATCH	4	2.81

MC 1122 Quantum

Sving væk joystickholder (voksen)
Swing-away joystick mount (adult)

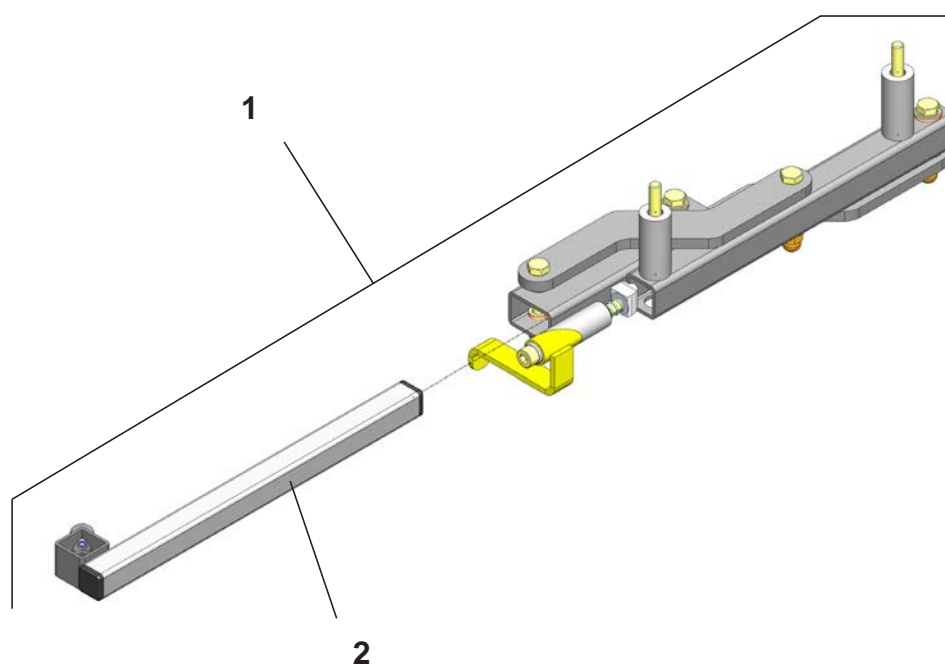




MC 1122 Quantum

Sving væk joysticksholder (voksen)
Swing-away joystick mount (adult)

Pos	Del/part no.	Beskrivelse	Part name	Antal/ quant.
1	2406-1037	Sving væk joysticksholder, højre, voksen	Swing-away joystick mount, right, adult	1
1	2406-1039	Sving væk joysticksholder, venstre, voksen	Swing-away joystick mount, left, adult	1
2	2406-1123	Justerbeslag 2, voksen komplet	Ajustable bracket 2, adult complete	1

MC 1122 Quantum

Armlæn
Armrests

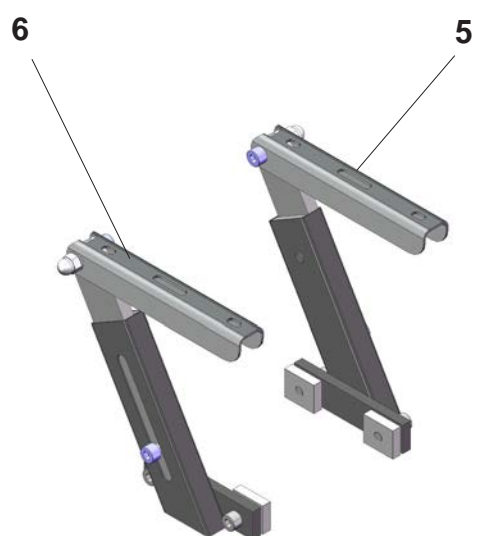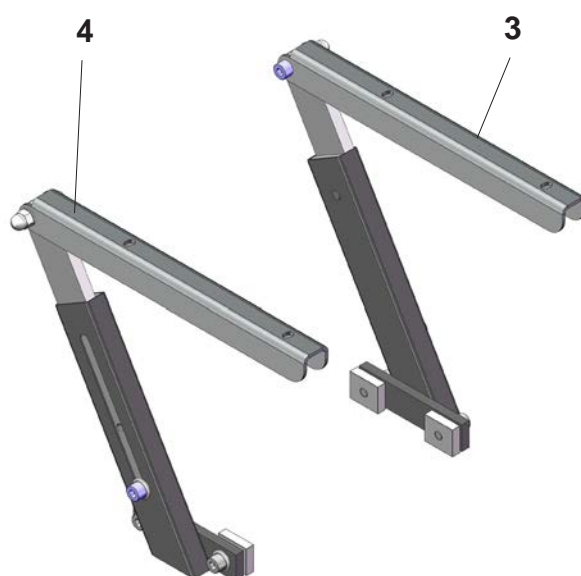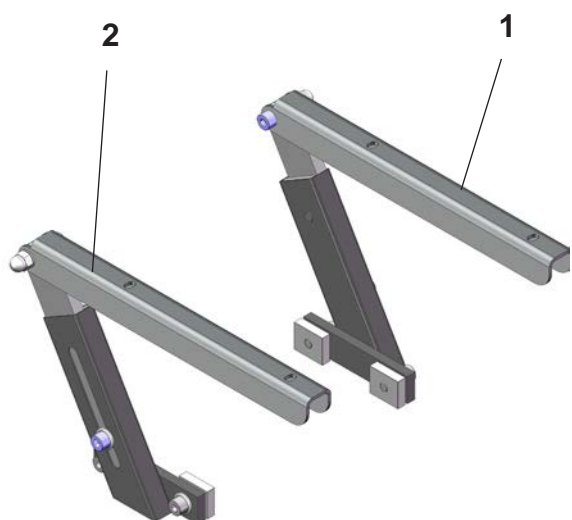

MC 1122 Quantum

Armlæn
Armrests

Pos	Del/part no.	Beskrivelse	Part name	Antal/ quant.
1	1507-1170	Armlænsholder lav, venstre, voksen	Armrest mount low, left, adult	1
2	1507-1169	Armlænsholder lav, højre, voksen	Armrest mount low, right, adult	1
3	1507-1195	Armlænsholder høj, venstre voksen	Armrest mount high, left adult	1
4	1507-1194	Armlænsholder høj, højre voksen	Armrest mount high, right adult	1
5	1507-1172	Armlænsholder, venstre junior	Armrest mount, left junior	1
6	1507-1171	Armlænsholder, højre junior	Armrest mount, right junior	1

